

CARESITE Micro™ Bifurcated Smallbore Extension Set

with removable CARESITE Micro Luer Access Device, Female Luer Lock Connector, and Spin-Lock® Connector

REF 354273

LENGTH 8 in. (20 cm) (approx.)

PRIMING VOLUME 0.5 mL (approx.)



STERILE EO Sterilized using ethylene oxide

 Single Use

 Do not resterilize

 Non-pyrogenic  Single sterile barrier system

 CAUTION

Not made with natural rubber latex or DEHP. Rx only A4809118-2 REV. 8/23

 **CAUTION: SET CONTAINS REMOVABLE LUER ACCESS DEVICE.**

Intended for direct injection, intermittent infusion, continuous infusion, or aspiration.

Instructions for Use: Use Aseptic Technique.

- Tighten luer access device before use.
- Prior to injection, close slide clamp on other leg to prevent upstream flow.
- **To access CARESITE Micro Luer Access Device:** Swab top of luer access device vigorously for 15 seconds with 70% Isopropyl Alcohol and allow to air dry prior to use. Use a luer connector without a needle. After medication administration, flush per institutional protocol.
- Replace set per CDC guidelines.

- Compatible with male luer connectors of internal diameter (ID) between 0.067 in. (1.7 mm) and 0.114 in. (2.9 mm).
- **For use with Power Injectors (maximum pressure of 300 psi and maximum flow rate of 10 mL/sec):** Secure luer access device with a luer lock connector. Refer to labeling of associate devices (e.g. catheters) for maximum pressure and flow rate to ensure compatibility.
 1. Prime extension set to ensure air is expelled.
 2. Firmly attach connectors to desired connections.

B. Braun Medical Inc.
Bethlehem, PA 18018-3524 USA
1-800-227-2862

B | BRAUN