

Filter Needle II

18 Ga. x 1.5 in. (3.8 cm) Needle and removable 5 μ m filter

REF 415043

STERILE EO

Sterilized using ethylene oxide

 Non-pyrogenic

 Single Use

 Single sterile barrier system

 Do not use if package is damaged and consult Instructions for Use





Do not resterilize.

Not made with natural rubber latex or DEHP.

Pictorials are for reference only.

Rx only

A4809176-2 REV. 11/24

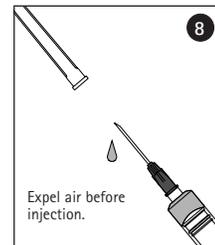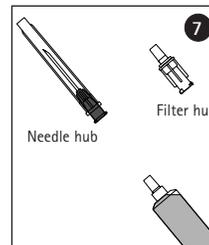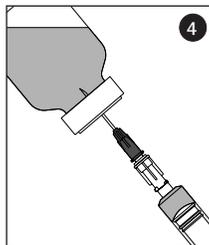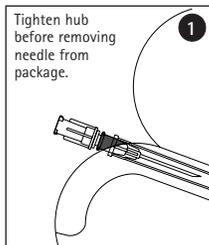

- For aspiration or injection of fluid after removal of filter
- **CAUTION: DO NOT ASPIRATE AND INJECT USING SAME NEEDLE.**

Instructions for Use: Use Aseptic Technique.

- Visually inspect sterile barrier system for breaches and integrity prior to use. Do not use if packaging or product is damaged or contaminated or if protective caps are loose or missing. Use of damaged or contaminated device might lead to injury, illness or death of the patient.
 - Replace needle per CDC guidelines.
1. Peel open package and expose white filter hub. Firmly seat filter hub into luer taper of attached needle hub, then remove entire device from package.

2. Grasp needle guard and firmly attach filter needle to luer lock fitting of syringe. The plastic needle guard serves as a wrench to attach and remove the filter needle.
3. Remove needle guard by pulling straight off. Do not twist.
4. Insert needle into rubber stopper of fluid container (cleanse site before insertion) and withdraw fluid through the filter needle into the syringe.
5. Remove syringe/filter needle from container and continue to aspirate until all fluid has been removed from needle and hub.
6. Replace guard over needle and twist to remove filter needle from syringe.
7. Twist to separate the filter hub (white) from the needle hub (color coded).
8. Discard filter hub and re-attach needle to filled syringe. Proceed with injection (expel air before injection).

B. Braun Medical Inc.

Bethlehem, PA 18018-3524 USA

1-800-227-2862

B | BRAUN