ENGLISH

Catheter Stabilization Device
Read carefully before use:
Safety and Efficacy Considerations:

⊗ Single use only. ⓧ Do not resterilize.

STERILE EO Sterile by Ethylene Oxide.

ⓧ Do not use if package is damaged.

Fluid path is nonpyrogenic (if present).

REF Product code, **LOT** lot number and

EXP expiration date information - see packaging.

Rx Only. Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Do not alter the STATLOCK® stabilization device or components. Procedure must be performed by trained personnel with knowledge of anatomical landmarks, safe technique and potential complications.

LATEX FREE

Indications for Use:

The STATLOCK® stabilization device is a stabilization device for compatible catheter hubs with push-tab.

Contraindications:

Known tape or adhesive allergies.

Warnings and Precautions:

1. Do not use the STATLOCK® stabilization device where loss of adherence could occur, such as with a confused patient or nonadherent skin.
2. Observe universal blood and body fluid precautions and infection control procedures, during application and removal of the STATLOCK® stabilization device.
3. SUTURE the STATLOCK® STABILIZATION DEVICE PAD TO SKIN IF DESIRED OR DEEMED NECESSARY.
4. Avoid contacting the STATLOCK® stabilization device with alcohol or acetone, both can weaken bonding of components and pad adherence.

5. Minimize catheter/tube manipulation during application and removal of the STATLOCK® stabilization device.
6. Remove oil and moisturizer from targeted skin area.
7. The STATLOCK® stabilization device pad adherence and catheter/tube position should be routinely inspected.
8. Orient the STATLOCK® stabilization device so arrow points toward catheter tip.
9. The STATLOCK® stabilization device should be replaced at least every 7 days.

Use sterile technique for application and removal of the STATLOCK® stabilization device.

See illustrations for specific details.

Application:

1. **PREP:** Prepare targeted area for the STATLOCK® stabilization device using alcohol, to remove oil and moisture, and skin prep for skin protection and enhanced adhesion. **ALLOW TO DRY COMPLETELY. ALWAYS CONNECT THE STATLOCK® STABILIZATION DEVICE TO CATHETER BEFORE ADHERING PAD TO SKIN.**
2. **PRESS:** Before applying the STATLOCK® stabilization device pad to skin, secure tube or catheter to STATLOCK® stabilization retainer.
3. **PEEL:** Peel off the STATLOCK® stabilization device paper backing, and
4. **PLACE:** Place the STATLOCK® stabilization device on targeted skin one side at a time.

Note:

- The STATLOCK® stabilization device pad can be adhered after X-ray confirmation of proper catheter/ tube position.

Removal:

Carefully remove overlying dressing using stretch technique and sutures (if applicable).

1. **DISSOLVE** adhesive with alcohol swab while gently lifting the STATLOCK® stabilization device pad.
2. **DISENGAGE** catheter/ tube from the STATLOCK® stabilization device retainer.
3. **DOCUMENT** the STATLOCK® stabilization device dressing change in the patient chart.
4. **DISPOSE** of all equipment in appropriate containers.

Manufacturer/Fabricante:
Bard Access Systems, Inc.
605 North 5600 West
Salt Lake City, UT 84116 USA
800-443-5505, 801-522-5000
www.bardaccess.com

Bard and StatLock are trademarks and/or registered trademarks of C. R. Bard, Inc. All other trademarks are the property of their respective owners.

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