



## Cardinal Health™ Kangaroo™ Gastrostomy Tubes

# Quick Reference Guide


Kangaroo™ is by your side

A gastrostomy is the surgical formation of an opening through the abdominal wall into the stomach. Often utilized to provide long-term nutritional support, feeding through a gastrostomy tube (g-tube) is an effective and convenient way to deliver nourishment to the body through an established stoma.

- Extended length of tubing allows for easier access to resident
- Extension sets are not needed when feeding through a g-tube
- Utilize Feed/Flush technology to help avoid tube occlusions



## Kangaroo™ Gastrostomy Feeding Tubes with Y-Ports

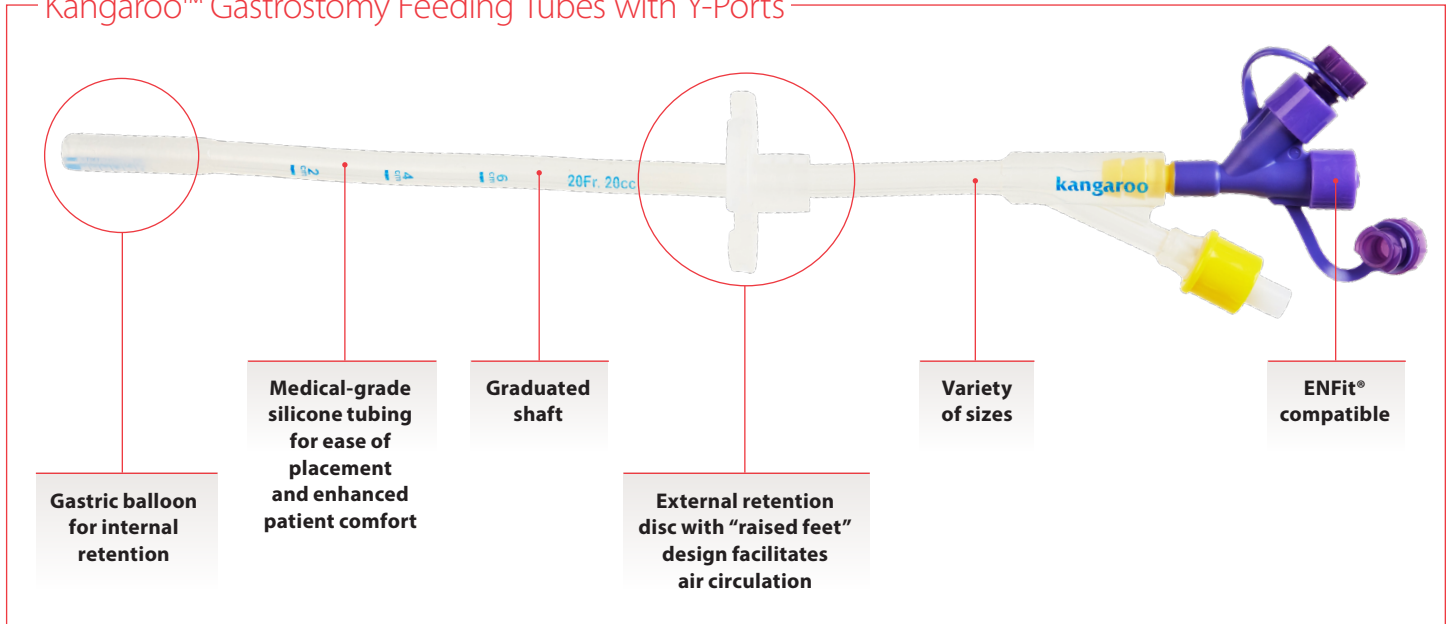

## Caring for residents with a g-tube

Caring for a g-tube is not considered a sterile procedure and can be incorporated into a normal daily caretaking routine. The skin around the tube should always be kept clean and dry to prevent the buildup of moisture and debris that may cause skin irritation and complications such as hypergranulation and infection.

### A daily check of the stoma site should include the following:

- Check the stoma site and skin for redness and swelling
- Check around the tube for excess skin growth
- Check the stoma site for frequent and/or large amounts of leakage

### Ordering information

ENFit® code	Non-ENFit® code	French size	Balloon size	Ship case
8884715122E	8884715122	12	5 cc	5
8884715148E	8884715148	14	5 cc	5
8884720163E	8884720163	16	20 cc	5
8884720189E	8884720189	18	20 cc	5
8884720205E	8884720205	20	20 cc	5
8884720221E	8884720221	22	20 cc	5
8884720247E	8884720247	24	20 cc	5
8884720265E	8884720265	26	20 cc	5
8884720285E	8884720285	28	20 cc	5

For more information or to order, contact your sales representative, call **877.CARDINAL** or visit **cardinalhealth.com/kangaroo**

