



avoset™

Quick Guide

Intermittent Delivery



Table of Content

Intermittent Infusion – Basic Functions

Review Preprogrammed Infusion Parameters	3
Review Infusion Parameters During Dose	4
Review Infusion Parameters During KVO (Between Doses).....	5
Reminder: Next Dose	6
Change the Next Dose Start Time.....	7
Time to change bag	8
Change Bag Now	9


i This quick guide provides guidelines for the basic functionalities of the Avoset infusion pump. If you are not a clinician please refer to the Important usage information guide, for additional information and the safe use of the product. For clinicians using the quick guide, please refer to the user manual. Quick guides, user manual and training materials terms of use can be found at www.eitanmedical.com



Review Preprogrammed Infusion Parameters




Press the *REVIEW* multi-function button to see the infusion parameters

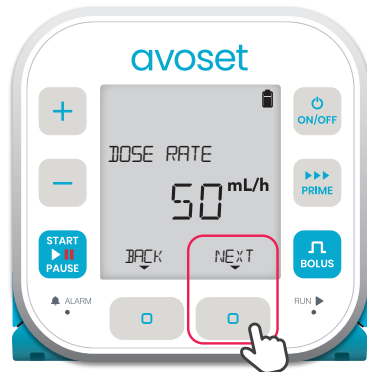

1st parameter:
The dose rate




2th parameter:
The volume to be infused in a single dose

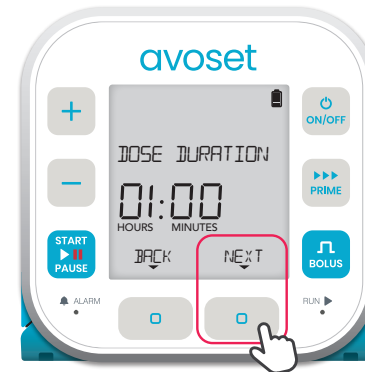


3th parameter:
The time required to complete a single dose




4th parameter:
The time between doses

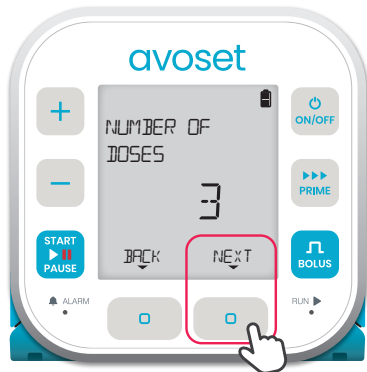

5th parameter:
The total number of doses to be delivered during the infusion

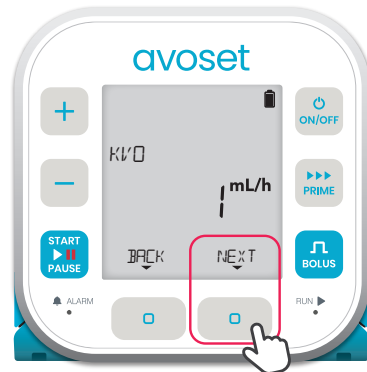

6th parameter:
The Keep Vein Open rate

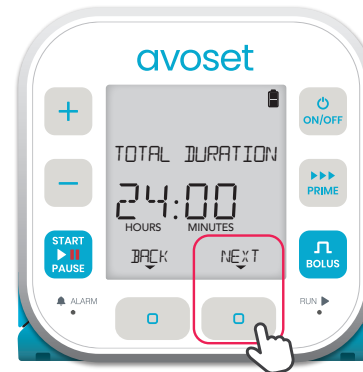


7th parameter:
The time required to complete the infusion



8th parameter:
The total volume to be infused during the infusion




Back to main display
(*REVIEW* option is available again)

 Based on the infusion programming, additional parameters may be displayed. Refer to the User Manual for further information.




Review Infusion Parameters During Dose

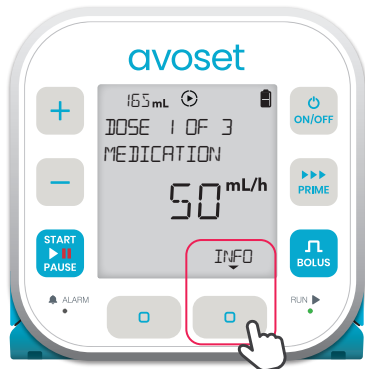


Press the **INFO** multi-function button to see the infusion parameters




1st parameter:
The time left to complete the current dose

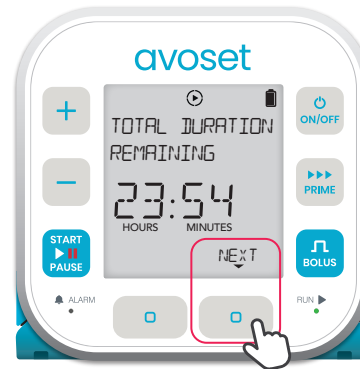

2nd parameter:
The time left to complete the infusion

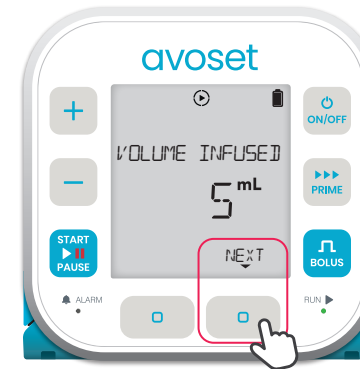

3rd parameter:
The total volume infused so far

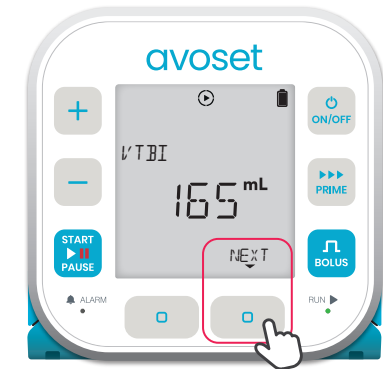


4th parameter:
The remaining volume to be infused




5th parameter:
The volume to be infused in a single dose



6th parameter:
The dose rate

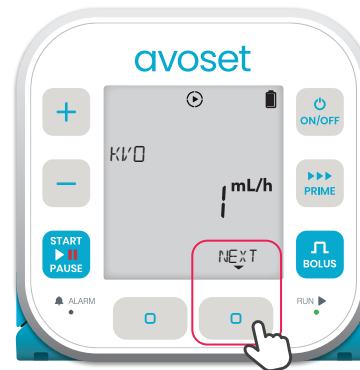


7th parameter:
The Keep Vein Open rate

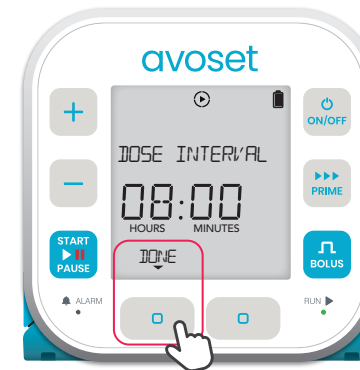

8th parameter:
The time between doses. Press the **DONE** multi-function button to navigate to the main display




Back to main display (**INFO** option is available again)

 Based on the infusion programming, additional parameters may be displayed. Refer to the User Manual for further information.



Review Infusion Parameters During KVO (Between Doses)



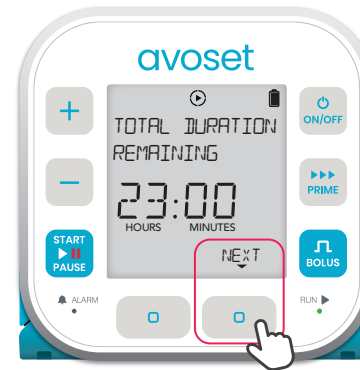


1st parameter:
The number of doses delivered so far out of the total doses

2nd parameter:
The time left before the next dose begins

3rd parameter:
The time left to complete the infusion

4th parameter:
The total volume infused so far

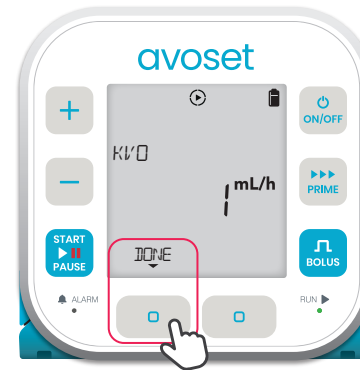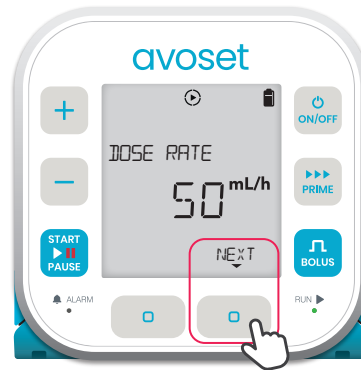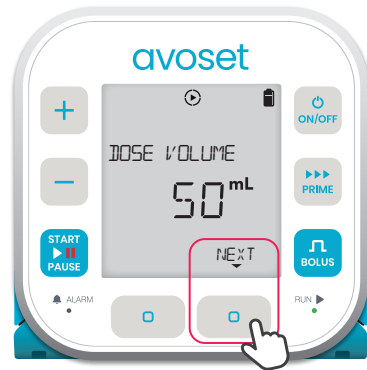

5th parameter:
The remaining volume to be infused

6th parameter:
The volume to be infused in a single dose

7th parameter:
The dose rate

8th parameter:
The Keep Vein Open rate. Press the DONE multi-function button to navigate to the main display

Back to main display (INFO option is available again)

i Based on the infusion programming, additional parameters may be displayed. Refer to the User Manual for further information.



Reminder: Next Dose


If the pump was set to remind you when the next dose is due, you will be prompted to resume the infusion manually if the pump was paused between doses:



1. NEXT DOSE IN notification shows how much time is left before the next dose should begin. Press the DISMISS multi-function button



2. DOSE DUE NOW notification is prompted when the next dose is supposed to begin. Press the DISMISS multi-function button

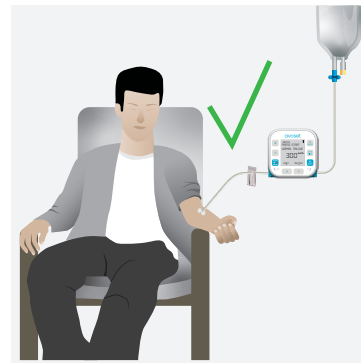

3. Verify the set is connected to the patient



4. Press START PAUSE to resume the infusion



Change the Next Dose Start Time


When the pump is paused Between Doses (during KVO), the time until the next dose begins can be shortened




1. Press **START PAUSE** to pause the infusion




2. Press the **EDIT** multi-function button to shorten the time remaining until the next dose starts

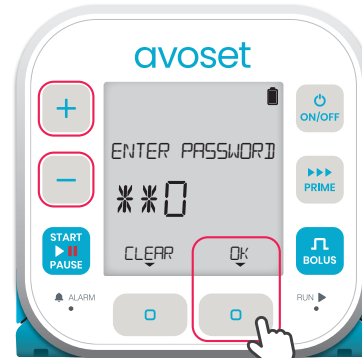

3. Password entry may be required. Use the **+** / **-** keys to set the password and press **OK**

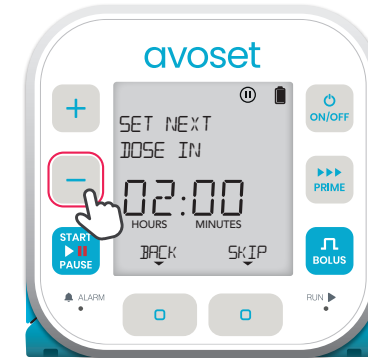

4. Use the **-** key to shorten the time remaining for the next dose

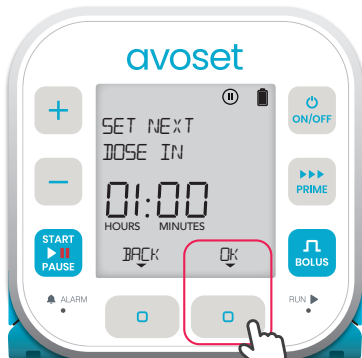

5. Press the **OK** multi-function button to set the selected time



6. Review the edited time, to confirm press **CONFIRM** to change press **EDIT**



7. Back to main display (the **INFO** option is available again) Press **START PAUSE** to resume



i By delivering this dose earlier, all remaining doses will also be delivered earlier.




Time to Change Bag

When the last dose has been delivered in full, the pump prompts you to change bag:



The pump notifies when the last dose in the bag was delivered

To continue infusion with a NEW BAG:

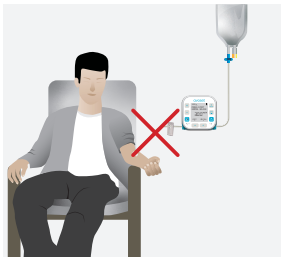

1. Disconnect set from patient




2. Push down the release latch and remove the used cassette and bag



3. Follow infusion set up on page 6



4. Press NEW BAG multi functional button

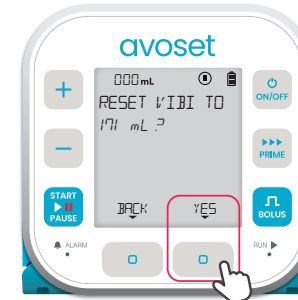

5. Press YES multi functional button




6. Press START PAUSE to resume the infusion, the dose will be given as scheduled

i For safe use, please make sure you complete setup process before moving to step 4.




Change Bag Now

If the bag has not yet been changed yet, and the next scheduled dose is up, the pump prompts to change bag now (immediately):

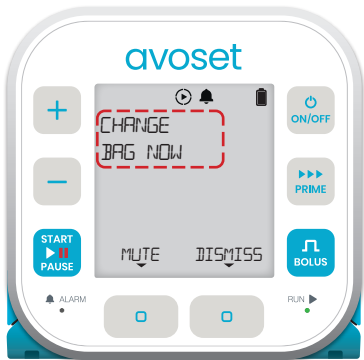

The pump notifies when the last dose in the bag was delivered

To continue infusion with a NEW BAG:

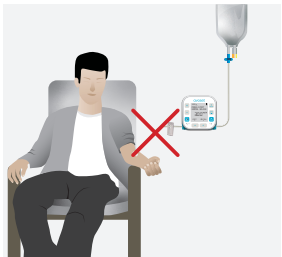

1. Disconnect set from patient



2. Push down the release latch and remove the used cassette and bag



3. Follow infusion set up on page 6



4. Press NEW BAG multi functional button



5. Press YES multi functional button



6. Press START PAUSE to resume the infusion, the dose will be given immediately

i For safe use, please make sure you complete setup process before moving to step 4.



This page is left intentionally blank



This page is left intentionally blank





Avoset English Quick Guide Intermittent | 15132-048-1018-UM Rev 1.0



MedNet EC-REP IIb GmbH
Borkstrasse 10
48163 Münster, Germany



Eitan Medical Ltd.
29 Yad Haruzim St.
P.O. Box 8639
Netanya 4250529, Israel



CE
0123

