



avoset™

Quick Guide


Patient PCA Delivery



Table of Content

PCA Infusion – Basic Functions

Review Preprogrammed Infusion Parameters	3
Review Infusion Parameters During Infusion	4
Deliver Patient Bolus	5

 This quick guide provides guidelines for the basic functionalities of the Avoset infusion pump. Please refer to the Important usage information guide, for additional information and the safe use of the product.




Review Preprogrammed Infusion Parameters



Press the *REVIEW* multi-function button to see the infusion parameters




1st parameter (optional):
The drug concentration

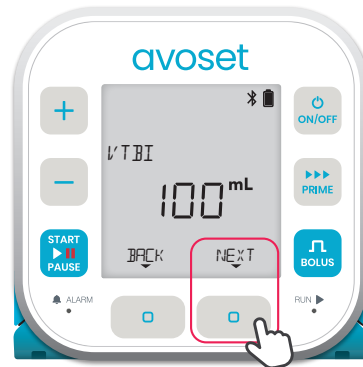

2nd parameter:
The total volume to be infused during the infusion



3rd parameter:
The infusion rate between boluses

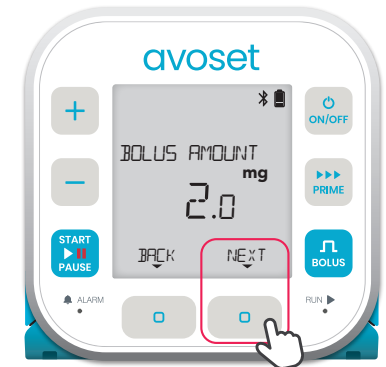

4th parameter:
The volume to be infused in a single bolus



5th parameter:
The time between boluses in which the user cannot initiate a bolus



Back to main display
(*REVIEW* option is available again)

i Based on the infusion programming, additional parameters may be displayed.



Review Infusion Parameters During Infusion




Press the **INFO** multi-function button to see the infusion parameters



1st parameter:
Time remaining until a bolus can be delivered.
*When a bolus is available, this parameter is not presented



2nd parameter:
The total volume infused so far

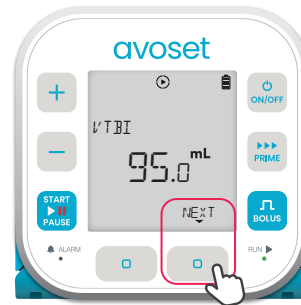


3rd parameter:
The remaining volume to be infused

Bolus History Parameters

4th parameter:
The time elapsed since bolus history was last cleared

5th parameter:
The number of requested boluses and given boluses

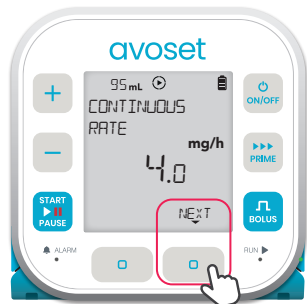

6th parameter:
Continuous rate



7th parameter:
The volume to be infused in a single bolus



8th parameter (optional):
The drug concentration
Press **DONE** to finish reviewing the parameters




Back to main display (**INFO** option is available again)

i Based on the infusion programming, additional parameters may be displayed.
Tip: long press on the **NEXT** multi-function button will return to home screen.



Deliver Patient Bolus



1. Press  to initiate a bolus



2. Bolus delivery indication

After delivering a bolus, a lockout period between boluses prevents the user from requesting additional bolus until the lockout period elapses.

Pressing the  key will prompt this notification



This page is left intentionally blank



This page is left intentionally blank





Avoset English Patient Quick Guide PCA | 15125-048-0007-UM, Rev.03



MedNet EC-REP IIb GmbH
Borkstrasse 10
48163 Münster, Germany



Eitan Medical Ltd.
29 Yad Haruzim St.
P.O. Box 8639
Netanya 4250529, Israel



CE
0123

