

Gebauer's  
**Pain Ease®**  
The **INSTANT** topical anesthetic.

## Instantly Control IV Start Pain

**Pain doesn't wait. Ease it in an instant. Prepare your patient for a more comfortable IV Start with Gebauer's Pain Ease® topical anesthetic skin refrigerant.**

- Temporarily controls pain and anxiety in as few as 4 to 10 seconds
- Use on intact skin, minor open wounds and intact oral mucous membranes
- May be used by any licensed healthcare practitioner without the order of a physician
- Nondrug, nonflammable formula



**Medium Stream Spray**

0386-0008-03

**Mist Spray**

0386-0008-02

*"Nurses rated the technical ease of the (IV) attempt as "easy" or "very easy" more often when the vapocoolant spray was used than when placebo was used..."*

*"Cannulation on first attempt was more often successful with the use of vapocoolant spray (85.0%) than with placebo (62.5%)"*

Farion K, Splinter K, Newhook K, Gaboury I, Splinter W. The effect of vapocoolant spray on pain due to intravenous cannulation in children: a randomized controlled trial. Canadian Medical Association Journal, July 1, 2008; 179 (1), pp 31-6.

### Important Risk and Safety Information for Gebauer's Pain Ease:

Published clinical trial results support the use in children three years of age and older. Do not use on large areas of damaged skin, puncture wounds, animal bites or serious wounds. Do not spray in eyes. Over spraying may cause frostbite. Freezing may alter skin pigmentation. Use caution when using product on diabetics or persons with poor circulation. Apply only to intact oral mucous membranes. Do not use on genital mucous membranes. The thawing process may be painful and freezing may lower resistance to infection and delay healing. If skin irritation develops, discontinue use. CAUTION: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

**Visit [gebauer.com/IVstart](http://gebauer.com/IVstart) to watch a video and learn how easy it is to bring instant relief to your patients' faces in 4 to 10 seconds flat.**

