# Operation Manual for the Pharm-Assist™ Dispensing Pump

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## Table of Contents

Introduction .....	2
What Comes With Your Purchase .....	2
Cautions and Warnings .....	2
Connections/Switches – Rear View of Pump .....	3
Keypad Functions .....	3 - 5
Memory – Storage, Recall, and Erase of Programs .....	5
Pump Settings .....	6
Tubing Sets .....	6
Connection of the tube to the Pharm-Assist™ .....	6
Priming the tubing .....	7
Alarms .....	7
Cleaning .....	7
Troubleshooting Guide .....	7
General Specifications .....	8
Pharm-Assist™ Automatic Labeler Reference Sheets .....	9 - 13
Pharm-Assist™ Dry Priming Instructions for Heavy Duty Tubing .....	14
Pharm-Assist™ Quick Start Guide .....	14 - 15
Warranty .....	15

## **Introduction**

The Pharm-Assist Dispensing Pump was designed by pharmacists to make the task of transferring fluids as easy, accurate, time-efficient, and cost-effective as possible. This has been achieved through the pump's versatile and user-friendly design.

The Pharm-Assist pump can increase efficiency in a variety of pharmacy functions that require the accurate dispensing of fluids. The following are a few examples of Pharm-Assist applications.

- Batching syringes (volumes as low as 0.1 mL in 3 mL syringes)
- Reconstitution of antibiotics (bulk drug vials or manufacturers' piggy back)
- Transferring large volumes in forward or reverse (up to 9,999 mL)
- Filling infusion devices (i.e.: Intermates, Homepumps, RediMeds, Pharmacia-Deltec cassettes, Mini-Bags, etc.)
- Transferring oral liquids
- Batching respiratory and anesthesia medications
- Batching Saline and Heparin flush syringes
- Printing syringe labels when used in conjunction with the Excelsior Thermal Labeler

## **What Comes With Your Purchase**

- Pharm-Assist Pump
- Foot Pedal
- Face Plate
- Power Cord
- Quick Start Guide
- Operations Manual

## **Cautions and Warnings**

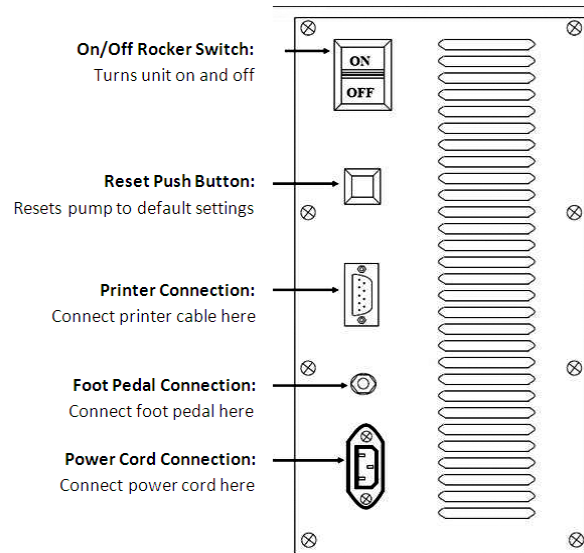
- Tubing set fluid path and area under protective end caps are sterile and non-pyrogenic in unopened and undamaged package. Use aseptic techniques with tubing set when removing caps, spiking diluent containers and making connections to luer adapter.
- The dispensing set should be changed within 24-hours due to touch contamination. DO NOT re-sterilize or reuse, it may cause damage to the tubing.
- Change the tubing set prior to each change of drug or incompatible fluid to avoid harmful interactions.
- DO NOT USE for intravenous administration or other routes of direct patient delivery.
- DO NOT exceed the maximum speed indicated for use with needles and filters.

<u>Needles or Filter</u>	<u>Maximum Speed</u>
16 G needle.....	99
18 G needle.....	35
Smaller needle than 18 G.....	5
0.2 micron filter.....	5
0.5 micron filter.....	10

- **ADJUST** may need to be used for a change in fluid viscosity or pump speed, use of needles, use of in-line filters, filling containers that create back pressure such as elastomeric or microbore tubing or a significant change in desired volume.
- Use of tubing sets with incorrect specifications may adversely affect volume accuracy, pump reliability and performance.
- DO NOT load or unload the tubing set with the power on or operate the pump without the blue protective plate in front of the rollers. Injury to fingers may occur.
- The power cord must be connected to a properly grounded hospital grade 120V receptacle for proper pump performance and safety.
- Fluid inside pump may cause damage and affect performance. Clean spills immediately.
- Use Heavy Duty “BX01”, “BX02”, or “BX03” tubing when filling Homepumps or other elastomeric pumps.
- Pump operation must be monitored at the beginning of each cycle and at intervals during the cycle to ensure the Pharm-Assist is operating within acceptable limits. Use ADJUST to achieve accuracy.
- Save the original box and packaging. In the event that the pump needs to be sent in for servicing, send the Pharm-Assist in its original packaging. If the original box cannot be located, contact Excelsior at 800-487-4276 and ask for a shipping box to be sent to you.
- Allow the pump to ventilate by keeping the air vents of the pump unobstructed.
- Service performed by persons other than Excelsior or its authorized agents, may, at the discretion of Excelsior, be cause to void the warranty at the end of this manual.

## Connections/Switches – Rear View of Pump

Cable connections and switch locations.

## Keypad Functions

To use each feature of the pump, simply press the numeric keys to enter the value followed by the function key. The programmed values will be displayed on the LCD's directly above or below the function keys.

### **NUMBER Keys:**

Used to enter numeric values prior to a function key. This method is used to reduce the required number of keystrokes.

### **CLEAR Key:**

Functions as a clear entry key, used to clear the current entry on the lower LCD prior to pressing a function key. It is also used to escape out of resume mode to abort an interrupted dispense cycle.

### **VOLUME Key:**

The default is 0 mL. This key is used to enter the desired volume to be dispensed by the pump. To enter a desired volume, simply press the numeric keys followed by **VOLUME**.

#### *Example:*

To enter a Volume of 50 mL, press 5, 0, and the **VOLUME** key. 50 mL will now be displayed in the Lower LCD above this key.

### **START/STOP Key:**

Used to start and stop the pump. The key can be interchangeably used with the foot pedal. The dispensing cycle may be interrupted by pressing this key. The pump will be in "resume mode" and "resume" will be displayed in the upper line of the top LCD. In this mode, the operator can continue the interrupted dispense cycle by pressing **START** or abort the cycle by pressing **CLEAR**.

#### *Example:*

The dispense cycle is programmed for 50 mL volume and the Pharm-Assist is stopped by pressing **STOP** or the foot pedal. The Pharm-Assist has dispensed only 35 mL of the 50 mL. The user can either press **START** to dispense the remaining 15 mL to complete the desired volume of 50 mL or press **CLEAR** to abort the cycle and be ready to begin a completely new dispense cycle of 50 mL.

**FOOT PEDAL:**

Interchangeable with the **START/STOP** key.

**ADJUST Key:**

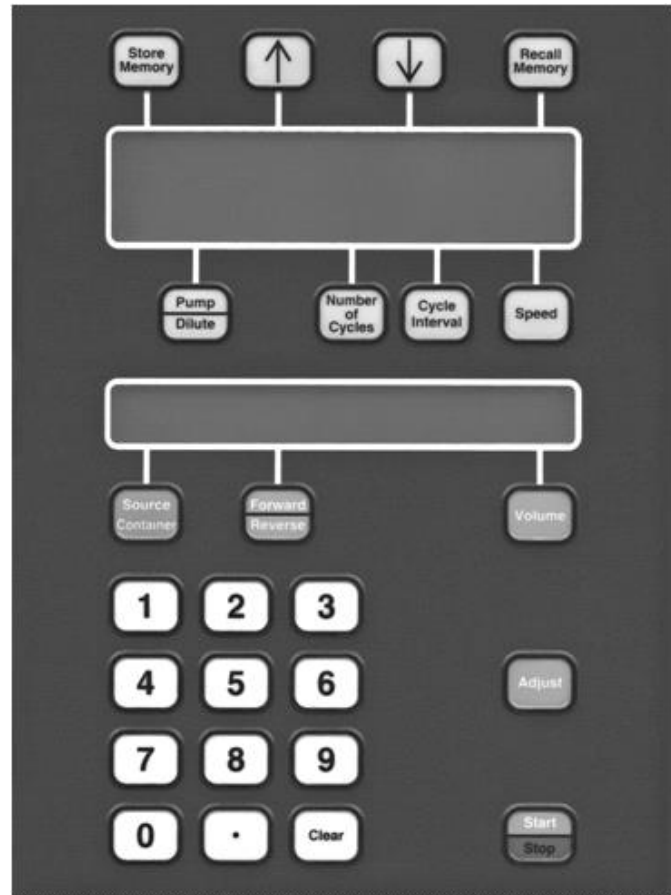
Use this key when the actual measured volume is different from the desired volume. Enter the actual measured volume by using the numeric keys followed by **ADJUST**.

The actual measured volume should always be checked with the desired volume. If the actual measured volume differs from the desired volume, **ADJUST** should be used. Verify the accuracy in a subsequent cycle, and if within specifications (+/- 1%), proceed with the programmed dispensing function.

It is possible to use **ADJUST** more than once if the previous entry was not precise in measurement or if the pump is not accurate. **ADJUST** must be used for each pumping application to achieve the desired accuracy level.

Any change in pumping parameters will necessitate additional use of this feature. Such parameter changes include, but are not limited to:

- a) change in fluid viscosity,
- b) change in pump speed,
- c) use of small bore size needles (less than 16 G),
- d) use of in-line filters,
- e) filling containers that create back pressure,
- f) significant change in **desired volume** and
- g) change in direction from forward to reverse.

*Example:*

The desired program volume is 50 mL. The measured volume is found to be 52 mL. Enter 5, 2 followed by **ADJUST**. The upper LCD displays “ADJUSTED” and the pump is now calibrated for 50 mL. The volume from the next pump cycle is measured and found to be exactly 50 mL. Note: Choose a convenient, but accurate method for measuring the actual volume. For example, you may verify pump accuracy by measuring: a) the output volume of the pump in a syringe, graduated cylinder, or by using a scale to verify the incremental weight of a bag or elastomeric or b) the reduction in volume or weight of the input source container such as a bag, graduated cylinder, syringe, bottle, or other receptacle.

**FORWARD/REVERSE Key:**

The default is forward. Forward will pump fluid from the spike end to the luer end.

Reverse will pump fluid from the luer end to the spike end. Pressing this key will alternate or toggle the pump between forward and reverse. “FWD” or “REV” is displayed on the lower LCD display directly above the key.

**SOURCE CONTAINER Key:**

The default is “off”. The feature will subtract all dispensed volumes pumped in forward operation from the amount entered as source container volume. Reverse operation has no effect on this feature. If there is not enough fluid in the source container to complete a dispense cycle, the pump will beep three times to alert the user. Pumping action will be prevented until the source container volume is increased so that it is equal to or greater than the desired volume; the desired volume is lowered to be equal to or lower than the source container volume; or this feature is turned off.

To use the feature, simply enter the initial source container volume by pressing the numeric keys followed by **SOURCE CONTAINER**. To turn off the feature, simply press **SOURCE CONTAINER** without pressing the numeric keys. The remaining volume in the source container will be displayed above this function key.

**SPEED Key:**

The default is speed 55. To change speed, press the numeric keys followed by **SPEED**. Minimum speed is 1 or approximately 150 mL/minute and maximum speed is 99 or approximately one liter per minute. The speed value will be displayed on the right side of the upper LCD display above the **SPEED** key.

Maximum speed guidelines for accuracy:

16 G needle = 99

18 G needle = 35

<18 G needle = 5

0.2 micron filter = 5

0.5 micron filter = 10

**NUMBER OF CYCLES Key:**

The preset is 1. This feature will automatically cycle the pump to complete the number of cycles entered with a selected interval in between each cycle. Enter the desired number of cycles by pressing the numeric keys followed by the **NUMBER OF CYCLES** key. The programmed number of cycles will be displayed directly above the key. The display will count down as each cycle is completed.

**CYCLE INTERVAL Key:**

The preset is 2 seconds. Entries can range from one to nine seconds between deliveries.

Enter the amount of seconds by pressing a numeric key followed by the **CYCLE INTERVAL** key. The programmed value will be displayed directly above the key. The display will count down between each dispensing cycle.

**PUMP/DILUTE Key:**

The preset is Pump. Pump is the normal setting. Dilute 1 and Dilute 2 will cause the rollers to reverse at the end of the cycle to pull back a drip of fluid that may occur as a result of filling devices with back pressure such as elastomerics.

Pump will pull back no fluid while Dil-1 and Dil-2 will pull back approximately 0.1 mL and 0.2 mL respectively. Pressing this key will toggle its value among these three settings. The setting will be displayed directly above the key.

**Memory – Storage, Recall, and Erase of Programs**

This feature is used to store up to 20 programs in memory that can be easily retrieved. The memory will store the desired volume, speed, pump/dilute and calibration. Use the store feature after all of these parameters have been set.

1. Set all the parameters on the pump displays (including calibration, volume, speed, and pump/dilute).
2. Select a memory location between 1 and 20 by pressing the numeric keys followed by **STORE MEMORY**. The memory location number will be displayed in the upper left of the top LCD. The location number will be followed by a blank and the letter "A".
3. Select a title for the stored program. Use the up and/or down arrows to change the "A" to the first character of the title. Letters, numbers, and a blank are available. When the desired character is displayed, press **STORE MEMORY**. The display will now have the first selected character followed by an "A".
4. Repeat step 3 for each of the characters. If the title is less than the full width of the display, press **RECALL MEMORY** to eliminate the last undesired "A".
5. The program is stored in memory and may be retrieved at any time by either pressing the memory location number followed by **RECALL MEMORY** or scrolling through all memory locations using the up and/or down arrow keys.
6. To erase a stored program from a memory location, either store another program into that location or press **STORE MEMORY** immediately after the program has been recalled.

**NOTES:**

The 0 memory location is the default pump settings; therefore, by recalling the 0 memory location, the pump is reset.

If saving adjusted volumes, be sure to check accuracy each time the program is recalled, as accuracy is sensitive to the setup of the pump. Any small change to the setup including using different needles or dispensing into different containers can cause this stored adjustment to fall out of acceptable limits.

If you desire the Pharm-Assist to automatically cycle a preset number of times, you may enter the number of cycles desired after recalling the program. The Number of Cycles cannot be stored.

## **Pump Settings**

Default Settings Table

Function	Default Setting
Volume	0.0 mL
Speed	55
Pump/Dilute	Pump mode
Forward/Reverse	Forward
Number of Cycles	1
Cycle Interval	2 seconds
Source Container	OFF

Valid Entries Table

Function	Valid Entries
Volume	0.10 to 9.95 mL increments of 0.05 mL 10.0 to 999.9 mL increments of 0.1 mL 1000 to 9999 mL increments of 1 mL
Speed	1 to 99 in increments of 1
Number of Cycles	1 to 999 in increments of 1
Cycle Interval	1 to 9 seconds in increments of 1 second
Source Container	1 to 9999 in increments of 1 mL

## **Tubing Sets**

Select the tubing set that best suits the needs of each given application.

PA01.....	<i>Single Lead Tubing Set</i>
PA02.....	<i>Double Lead Tubing Set</i>
PA03.....	<i>Triple Lead Tubing Set</i>
PA01LL.....	<i>Single Lead Tubing Set, Double Luer</i>
BX01.....	<i>Single Lead Tubing Set, Heavy Duty</i>
BX02.....	<i>Double Lead Tubing Set, Heavy Duty</i>
BX03.....	<i>Triple Lead Tubing Set, Heavy Duty</i>
BX01LL.....	<i>Single Lead Tubing Set, Heavy Duty, Double Luer</i>
LPA01.....	<i>For Fat Emulsion Dispensing</i>

NOTE: If you find that tubing sets are getting damaged or delivering inaccurate volumes, lower the speed of operation or switch to heavy duty tubing sets.

## **Connection of the tube to the Pharm-Assist™**

Refer to the pictures below.

1. Turn roller to help install tubing.

2. Tubing should be on ridge of roller.

## **Priming the tubing**

1. Connect the spike end of the tubing set to the diluent and place the needle into a suitable waste container.
2. Remove the air from the dispensing set or prime. Run the pump until all of the air bubbles are removed. (approx. 27 mL of diluents to prime the PA01 tubing set)

NOTE: An accurate dose cannot be achieved with air in the dispensing set.

## **Alarms**

**Dispensing cycle completion:** One beep will sound at the end of each cycle to let the user know that the pump has completed the dispensing cycle.

**Incorrect Key Sequence:** One beep will sound to indicate an incorrect keystroke sequence. For example, the pump will not allow an entry of greater than 999 for number of cycles.

**Zero Entry:** Two beeps will sound to alert the user that the entry was not accepted and the lowest value is displayed. Number of cycles, cycle interval, and speed will display one.

**Maximum Value Exceeded:** Two beeps will sound to alert the user that the entry was not accepted and the highest value is displayed. Speed will display 99 and cycle interval will display 9.

**Source container:** Three beeps will sound to indicate the source container volume is less than the desired volume.

## **Cleaning**

Immediately turn off the power to the pump. Wipe up fluid spills and splashes immediately to prevent entry into the pump. Use lint-free cloths or swabs only with warm water, isopropyl alcohol, or mild solvents. Do not use any abrasives. Also clean the inside of the Pharm-Assist, behind the blue front plate, including the rollers with the power **OFF**.

## **Troubleshooting Guide**

Silicone tubing herniates or balloons and prevents pumping of fluid:	a.) Filling Homepumps or other elastomeric pumps; use Heavy Duty tubing (BX series) b.) Using smaller than 16 G needle; switch to 16 G needle or significantly reduce the speed of the pump to reduce the back pressure. c.) Using inline filter; significantly reduce speed to reduce back pressure.
Tubing kicks out of pump:	a.) Too much back pressure for the speed (use of needles, filters or filling device). Decrease back pressure or significantly lower speed. b.) Tubing was not installed properly.
Inaccurate filling, but repeatable (consistently wrong):	a.) Use the <b>ADJUST</b> key.
Inaccurate filling, not repeatable (varies all over), for example, due to unequal back pressure because of 18 G needle or filter:	a.) Lower the speed.
Does not pump and makes loud grinding noise; tubing is herniated and preventing rollers from moving:	a.) Decrease back pressure (use larger needle or remove filter). b.) Lower the speed. c.) Use "BX" series tubing.
Pump displays are dark:	a.) Power cord not connected.

## General Specifications\*

<b>Delivery Rate</b>	3 mL/sec to 16 mL/sec (approximately). This may vary slightly and is determined by pump speed, fluid viscosity, and back pressure of in-line devices or filling receptacle.
<b>Preset Operating Settings</b>	Forward dispensing (from spike to luer), Speed of 55, one cycle, and 0.0 mL volume.
<b>Allowed Volume Entries</b>	0.1 mL to 9,999 mL
<b>Dispensing Volume Accuracy</b>	+/- 1% (using Adjust feature)**
<b>Electrical Requirements</b>	120 VAC/60HZ
<b>Weight</b>	16 lbs. (approximate)
<b>Operating Temperature</b>	45°F - 85°F (7 – 29° C)
<b>Storage Capacity for Programs</b>	20 Stored programs

\*Tubing sets with Excelsior specifications ensure accuracy and decrease user variability. Excelsior's set contains a bar connector which fits into a slot on the Pharm-Assist pump. The bar connector keeps the tubing seated properly and decreases user variability in installation.

\*\*The actual measured volume should always be compared with the desired volume. If the actual measured volume differs from the desired volume, **ADJUST** should be used. Verify the accuracy in a subsequent cycle, and if within specifications (+1/-1%), proceed with the programmed dispensing function. Any change in pumping parameters will necessitate additional use of this feature. Such parameter changes include, but are not limited to:

- a) Change in fluid **viscosity**.
- b) Change in pump **speed**.
- c) Use of **needles** with a smaller bore size than 16 G.
- d) Use of in-line **filters**.
- e) Filling containers that create back pressure such as **elastomerics** or **microbore** tubing
- f) Significant change in desired **volume**.
- g) Change from forward to **reverse**.

## **Pharm-Assist™ Automatic Labeler Reference Sheets**

(Use in conjunction with the LP 2824 Thermal Printer Quick Start Guide)

### **Contents:**

1. Must Read the LP 2824 Thermal Printer Quick Start Guide
2. Installing Labels in the Printer
3. Quick Start for the Automatic Labeler
4. Correcting Label Jams in the Printer
5. Custom Label Procedure
6. Custom Label Examples
7. Drug Selection Listing

### **1. Must Read the LP 2824 Thermal Printer Quick Start Guide**

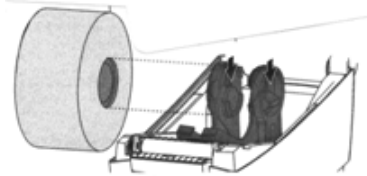
### **2. Installing Labels in the Printer**

- 2.1. **TURN OFF THE POWER TO THE PRINTER.**
- 2.2. Load media.


Open Cover

Remove the remaining labels and its roller inside of the printer.

Sample Roll of Labels

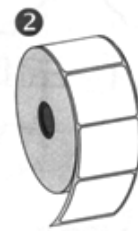
Adjust Holders

Place Roll into Printer

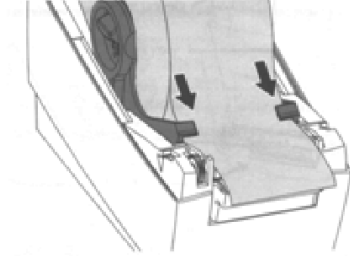Thread Media through Guides

Close Cover

### **3. Quick Start for the Automatic Labeler**

- 3.1. Plug in the Pharm-Assist and the Thermal Printer.
- 3.2. Connect the RS-232 Serial Cable (Nine pins Sub-D connector at each end) into the back of the Pharm-Assist and Thermal Printer.
- 3.3. Turn on the Pharm-Assist and Thermal Printer.
- 3.4. Enter Label Mode of the Pharm-Assist by pressing 100 on the keypad followed by **RECALL MEMORY** key. The bottom line of the upper LCD display will prompt you to "SELECT DRUG". The lower LCD display will have a drug name.
- 3.5. **SELECT THE DRUG:**
  - 3.5.1. Enter the drug number (from the drug listing sheet) followed by **RECALL MEMORY** key or use the up and down arrows of the keypad to scroll through the drug choices.
  - 3.5.2. Press the **VOLUME** key to enter the drug selection.
- 3.6. **ENTER THE DOSE:**

- 3.6.1. The lower LCD display will have a default setting for the dose of the drug that you have just selected. If the dose is correct, press the **VOLUME** key. If the dose is incorrect, enter the correct dose by using the numeric keypad followed by the **VOLUME** key.

**Note:** The default units of grams or milligrams cannot be changed.

3.7. ENTER THE CONCENTRATION:

- 3.7.1. The lower LCD display will have a default setting for the concentration of the drug that you have just selected. If the concentration is correct, press the **VOLUME** key. If the concentration is incorrect, enter the correct concentration by using the numeric keypad followed by the **VOLUME** key.

3.8. ENTER THE DATE:

- 3.8.1. Enter the date by using the numeric keypad followed by the **VOLUME** key. You can enter a slash to separate the numbers by using the **DECIMAL POINT** key on the numeric keypad.

3.9. ENTER THE LOT NUMBER:

- 3.9.1. Enter up to a fourteen digit lot number by using the numeric keypad followed by the **VOLUME** key. Press the **DECIMAL POINT** key on the keypad to enter a “-” in the lot number.

3.10. ENTER THE BARCODE:

- 3.10.1. Enter up to a fifteen digit barcode by using the numeric keypad followed by the **VOLUME** key. **Do not** place a “-” in the barcode by pressing the **DECIMAL POINT** key on the numeric keypad.

3.11. ENTER THE NUMBER OF LABELS TO PRINT:

- 3.11.1. Enter the number of labels to print by using the numeric keypad followed by the **VOLUME** key.

3.12. START THE PRINTING PROCESS:

- 3.12.1. Press the **START/STOP** key on the keypad.

**NOTES:**

- A. Each press of the **STORE MEMORY** key will return to the previous prompt in Label Mode.
- B. The **PUMP/DILUTE** key will escape from Label Mode back to the last set-up of the pump prior to entering Label Mode. This will maintain the calibration and other programmed settings of the pump.

The Reset Button or the On/Off button of the pump will both put the pump into the Initial default settings, which occur when the pump is originally turned on. **IF YOU SHUT OFF POWER OR RESET THE PUMP, THE CALIBRATION AND OTHER PROGRAMMED SETTINGS OF THE PUMP WILL BE RESET TO THE DEFAULT SETTINGS.**

- C. The label printing may be stopped by turning off the printer.

**4. Correcting Label Jams in the Printer**

**CAUTION: TURN POWER OFF TO THE PRINTER BEFORE MAINTENANCE AND DO NOT UNDER ANY CIRCUMSTANCE, PLACE ANY SHARP, HARD OR ABRASIVE OBJECT SUCH AS A KNIFE OR SCREWDRIVER INTO THE PRINTER TO REMOVE A LABEL!!**

- 4.1. Open the printer cover which is located on top of the printer by sliding green Cover Release forward on each side of the printer to unlatch it and then tilt it upward to open.
- 4.2. Carefully remove the jammed/stuck label(s) if possible and the roll of labels if needed.
- 4.3. If needed follow the procedure for cleaning the thermal print head in LP2824 Thermal Printer Quick Start.
- 4.4. If needed follow the procedure for cleaning the Platen in LP2824 Thermal Printer Quick Start.

**CAUTION: DO NOT USE ANY ABRASIVE MATERIAL; IT MAY DAMAGE THE PRINTING MECHANISM!!**

**5. Custom Label Procedure**

5.1. ENTER CUSTOM LABEL MODE:

- 5.1.1. With the Pharm-Assist in the pumping mode (not the label mode) enter 1, 0, 1 followed by **RECALL MEMORY**.

5.2. SELECT A LABEL MEMORY LOCATION:

- 5.2.1. Use the arrow keys to scroll through the label memory locations. To add a customized label, scroll to the first available “USER” label and press **ADJUST**. To edit an existing customized label, scroll to the desired custom label and press **ADJUST**.
- 5.3. ENTER DRUG NAME (or one line message):
  - 5.3.1. The letter 'A' will appear in the first column. Use the up and/or down arrows to change the 'A' to the first character of the drug name. When the desired character is displayed, press **STORE MEMORY**. The display will now have the first selected character followed by an 'A'.

Use the arrow keys to select the second character and press **STORE MEMORY**. Continue until the entire drug name is displayed. If the drug name is less than 16 characters, press **RECALL MEMORY** to eliminate the last undesired 'A'.
- 5.4. SELECT DOSE UNITS:
  - 5.4.1. Use the arrow keys to scroll through the dose units. When the desired unit is displayed, press **STORE MEMORY**. You will not be able to enter a dose in step 5 if you do not select dose units.
- 5.5. ENTER DOSE:
  - 5.5.1. Type in the dose by using the numeric keypad followed by **STORE MEMORY**. You will not be able to enter a dose if you did not select dose units in Step 4.
- 5.6. SELECT CONCENTRATION UNITS:
  - 5.6.1. Use the arrow keys to scroll through the concentration units. When the desired concentration units are displayed, press **STORE MEMORY**. You will not be able to enter a concentration in step 7 if you do not select concentration units.
- 5.7. ENTER CONCENTRATION:
  - 5.7.1. Type in the concentration by using the numeric keypad followed by **STORE MEMORY**. You will not be able to enter a concentration if you did not select concentration in step 6.
- 5.8. ENTER VOLUME:
  - 5.8.1. Type in the volume, in milliliters, using the numeric keypad followed by **STORE MEMORY** (If you do not enter a volume, none will print). All volumes entered will print with “mL” units.
- 5.9. EXIT:
  - 5.9.1. Press **PUMP/DILUTE** at the “Select label” prompt to return to pump mode.

Note: Once a custom label is entered, it can be modified or eliminated by storing a different custom label at its memory location.

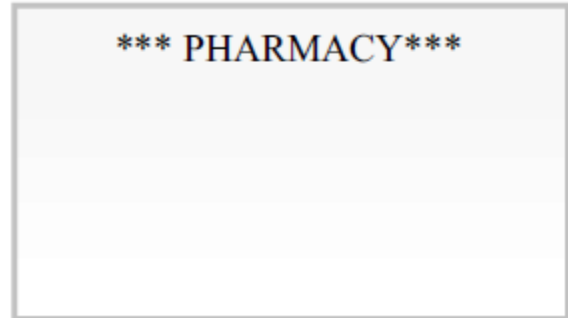
## 6. Custom Label Examples

You can create a wide variety of labels, with just a little planning. Here are three examples to get you started.

### Example #1

\*\*\*PHARMACY \*\*\*

1. For the drug name enter: “\*\*\* PHARMACY”
2. For the dose units select: “\*\*\*”
3. Select “ ” for the concentration units.
4. Do not enter anything for the volume.
5. When printing if you do not enter a date, lot number, and bar codes only the one line with \*\*\* PHARMACY \*\*\* will print.


---

### Example #2

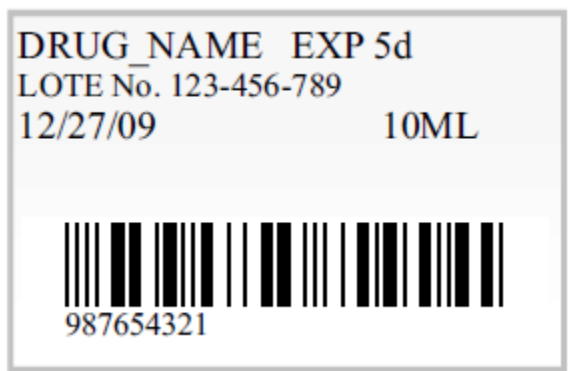DRUG\_NAME EXP 5d  
LOT# 123-456-789

1. For the drug name enter “DRUG\_NAME\_EXP”.
2. For the dose units select “d”.
3. For the dose enter “5”.
4. For the concentration units select “ ”
5. For the volume enter “10”.
6. When printing enter “12/27/09” for the date, “123-456-789” for the lot number, and “987654321” for the bar code.


---

### Example #3

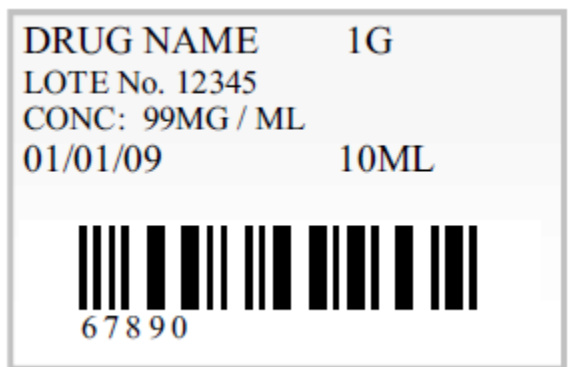DRUG\_NAME 1G  
LOT# 12345  
CONC: 99MG/ML

1. For the drug name enter “DRUG\_NAME 1G”. (The “1” will not print, but will display on the pump to identify the dosage.)
2. For the dose units select “G”.
3. For the dose enter “1”.
4. Select Conc. Units: “MG/ML”.
5. For the Conc. Enter “99”.
6. For the volume enter “10”.
7. When printing, enter a date of “01/01/09”, a lot number of “12345”, and a bar code of “67890”.

Call Excelsior if you have any problems or questions.  
800-4-U-Pharm / 800 487-4276 / 732 776-7525

7. Drug Listing for Zebra Printer

12/2/05

1.	Refg.Exp.date/tie		57.	Clindamycin	300mg	113.	Tobramycin	90mg
2.	Amikacin	250mg	58.	Clindamycin	600mg	114.	Tobramycin	100mg
3.	Amikacin	300mg	59.	Clindamycin	900mg	115.	Tobramycin	120mg
4.	Amikacin	450mg	60.	Cloxacillin	1G	116.	Tobramycin	140mg
5.	Amikacin	500mg	61.	Cloxacillin	2G	117.	Tobramycin	160mg
6.	Ampicillin	250mg	62.	Dilantin	100mg	118.	Vancomycin	500mg
7.	Ampicillin	500mg	63.	Dilantin	300mg	119.	Vancomycin	1G
8.	Ampicillin	1G	64.	Dilantin	500mg	120.	Zefazone	1G
9.	Ampicillin	1.5G	65.	Doxycycline	100mg	121.	Zefazone	2G
10.	Ampicillin	2G	66.	Erythromycin	500mg	122.	Zofran	8mg
11.	Amp.Sulbactam	1.5G	67.	Erythromycin	1G	123.	Zofran	10mg
12.	Amp.Sulbactam	3G	68.	Ethacrynate	50mg	124.	Zofran	12mg
13.	Azlocillin	2G	69.	Famotidine	20mg	125.	Zosyn	4.5G
14.	Azlocillin	3G	70.	Furosemide	40mg	126.	Zosyn	2.25G
15.	Azlocillin	4G	71.	Furosemide	1G	127.	Zosyn	3.375G
16.	Aztreonam	500mg	72.	Gentamicin	40mg	128.	Normal Saline	3 mL
17.	Aztreonam	1G	73.	Gentamicin	50mg	129.	Normal Saline	5 mL
18.	Aztreonam	2G	74.	Gentamicin	60mg	130.	Normal Saline	10 mL
19.	Cefamandole	1G	75.	Gentamicin	70mg	131.	Sterile Water	
20.	Cefamandole	2G	76.	Gentamicin	80mg	132.	Anectine	
21.	Cefazolin	500mg	77.	Gentamicin	90mg	133.	Atropine	
22.	Cefazolin	1G	78.	Gentamicin	100mg	134.	Brevital-SW	
23.	Cefazolin	1.5G	79.	Gentamicin	120mg	135.	Curare	
24.	Cefazolin	2G	80.	Gentamicin	140mg	136.	Ephedrine-NS	
25.	Cefepime	1G	81.	Gentamicin	160mg	137.	Heparin Flush	
26.	Cefepime	2G	82.	Gentamicin	180mg	138.	Morphine-PCA	
27.	Cefmetazole	1G	83.	Methyldopa	250mg	139.	Pentothal	
28.	Cefmetazole	2G	84.	Methyldopa	500mg	140.	Prostigmine	
29.	Cefonicid	1G	85.	Metoclopramide	5mg	141.	Recombivax	
30.	Cefonicid	2G	86.	Metoclopramide	10mg	142.	Robinul	
31.	Cefoperazone	1G	87.	Metoclopramide	20mg	143.	Versed (5 mg/mL)	
32.	Cefoperazone	2G	88.	Metoclopramide	30mg	144.	Versed (1 mg/mL)	
33.	Cefotaxime	1G	89.	Metoclopramide	50mg	145.	*** STAT ***	
34.	Cefotaxime	2G	90.	Mezlocillin	1G	146.	*** REFRIGERATE ***	
35.	Cefotetan	1G	91.	Mezlocillin	2G	147.	***DO NOT SHAKE	
36.	Cefotetan	2G	92.	Mezlocillin	3G	148.	USE NORMAL SPEED	
37.	Cefoxitin	500mg	93.	Mezlocillin	4G	149.	USE SLOW SPEED	
38.	Cefoxitin	1G	94.	Nafcillin	1G	150.	USE X-SLOW SPEED	
39.	Cefoxitin	2G	95.	Oxacillin	1G	151.	CENTRAL LINE ONLY	
40.	Ceftazidime	500mg	96.	Oxacillin	2G	152.	INFUSE> 60 MIN	
41.	Ceftazidime	1G	97.	Penicillin G	0.5 MU	153.	USER_A *	
42.	Ceftazidime	2G	98.	Penicillin G	1 MU	154.	USER_B *	
43.	Ceftizoxime	1G	99.	Penicillin G	1.5 MU	155.	USER_C *	
44.	Ceftizoxime	2G	100.	Penicillin G	2 MU	156.	USER_D *	
45.	Ceftriaxone	500mg	101.	Penicillin G	2.5 MU	157.	USER_E *	
46.	Ceftriaxone	1G	102.	Primaxin	250mg	158.	USER_F *	
47.	Ceftriaxone	2G	103.	Primaxin	500mg	159.	USER_G *	
48.	Cefuroxime	750mg	104.	Ranitidine	50mg	160.	USER_H *	
49.	Cefuroxime	1.5G	105.	Ticarcillin	1.5G	161.	USER_I *	
50.	Cephalothin	1G	106.	Ticarcillin	3G	162.	USER_J *	
51.	Cephalothin	2G	107.	Timentin	3.1G	163.	USER_K *	
52.	Cephapirin	1G	108.	Tobramycin	40mg	164.	USER_L *	
53.	Cephapirin	2G	109.	Tobramycin	50mg	165.	USER_M *	
54.	Chloramphenicol	1G	110.	Tobramycin	60mg	166.	USER_N *	
55.	Cimetidine	300mg	111.	Tobramycin	70mg	167.	USER_O*	
56.	Cimetidine	450MG	112.	Tobramycin	80mg			

## **Pharm-Assist™ Dry Priming Instructions for Heavy Duty Tubing**

When using BX01 tubing with the Pharm-Assist it is necessary to prime the pump in order to assure pumping accuracy and proper pump function.

1. Remove the dispensing set from its bag. Do not remove the protective caps from either end of the dispensing set.
2. While the power is off, take the blue pump face-cover off by lifting it up and towards you.
3. With the red bar lock tie (spike end) on top, place the connector of the tubing set into the slot on the right of the roller head. Push the connector in as far as it will go, and then take your fingers and rotate the roller head to seat the silicone tubing in the center of the groove of the pump housing.
4. Rotate the roller head a few times to loosen the silicon of the tubing.
5. Turn the power to the unit on and program a low speed of 6 and volume 100 and press **START**. Allow the unit to run for 30 seconds.
6. The tubing is now primed and ready to be used.

If you have any further questions or need additional assistance, please call our Customer Service Department at (800) 487-4276.

## **Pharm-Assist™ Quick Start Guide**

1. **POWER OFF**
2. **REMOVE THE FACE PLATE** from the roller head assembly
3. **TUBING SET INSTALLATION** – see pages 6-7 of user manual;
  - a) Insert the white bar connector into the slot located on the right side of the roller head.
  - b) Make sure that the red plastic tie at the silicone connection is located on the top side of the slot and that the white bar connector is pushed all the way into the slot.
  - c) Rotate the roller head manually to feed the silicone portion of the dispensing set into the roller head assembly.
4. **SPIKE** the bag/bottle of diluent/drug to be dispensed.
5. **POWER ON**
6. **PROGRAM TO PRIME**
  - a) Use the numeric key pad to enter 27
  - b) Press the **VOLUME** function key. The volume of 27 mL to prime the tubing set will be indicated on the lower LCD display.

**NOTE:** It takes 27 mL to prime single lead dispensing sets.

7. **PRIME.** Have a waste receptacle available. Press the **START** function key. REMOVE ALL OF THE AIR FROM THE TUBING SET.
8. **PREPROGRAMMED DEFAULTS.** The Pharm-Assist pump will default to a speed of 55 and one cycle when the pump is powered on. You may use the defaults or enter your own settings.
  - a) To Change the speed setting; Use the numeric keypad to enter the speed value, then press the **SPEED** function key.
  - b) TO USE CYCLE MODE;
    - i) Use the numeric keypad to enter the number of desired cycles and then press the **NUMBER OF CYCLES** function key.
    - ii) Use the numeric keypad to enter the interval of time between filling cycles (1-9 seconds), then press the **CYCLE INTERVAL** key.
9. **PROGRAM TO FILL BAXTER INTERMATES**
  - a) Use the numeric keypad to enter the desired volume in milliliters, then press the **VOLUME** key.
  - b) Press the **PUMP/DILUTE** function key until you see DIL-2 on the upper LCD display. This will pull back any drip that may occur at the end of the tubing set due to back pressure.
10. **FILLING BAXTER INTERMATES.** Attach the Intermate to the luer connection on the tubing set. Press the **START** function key. The pump will beep at the completion of a filling cycle. If in **CYCLCE MODE**, the screen will count down the seconds between the end of the one cycle and the start of the next cycle.
11. **CHECK ACCURACY.** If the Pharm-Assist is not within an acceptable accuracy range, use the **ADJUST** function key.
  - a) Enter the amount of volume actually dispensed using the numeric keypad, then press the **ADJUST** function key.

**NOTE:** For Intermates, we recommend that you use a scale to check accuracy. Weigh the Intermate before and after filling and then compare the actual weight of fluid dispensed with the expected weight of fluid dispensed. Be sure to account for the specific gravity of the fluid when determining the expected weight.

12. **CHECK FOR ACCURACY AFTER ADJUSTING.** Press the **START** function key to fill the Intermate. Your pump should be within the acceptable accuracy range of +/- 1% if adjusted properly.

Call Excelsior if you have any problems or questions.  
800-4-U-Pharm / 800 487-4276 / 732 776-7525

## **Warranty**

The Pharm-Assist pump is warranted against defects in material and workmanship for a period of twelve months from the date of delivery. Excelsior will repair or replace, at its option, any product that proves to be defective during the warranty period, provided proper use and maintenance procedures have been followed as described in the operation manual. If upon examination, Excelsior determines that misuse of product is the cause for the repair, all labor, material, and shipping costs involved shall be paid by the Buyer.

All defective products or components shall be returned to Excelsior with a detailed explanation of the failure. Buyer shall obtain a return authorization prior to return of product and all transportation charges must be prepaid.

**Limitations:** THE PROVISIONS ABOVE CONSTITUTE EXCELSIOR'S SOLE OBLIGATION AND EXCLUDE ALL OTHER REMEDIES OR WARRANTIES EXPRESS, STATUTORY OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, whether or not purposes or specifications are described herein. Excelsior further disclaims any responsibility whatsoever to Buyer or to any other person for injury to person or damage to or loss of property or value caused by any product that been subjected to misuse; negligence, or accident; or misapplied or used in violation of product manuals, instructions or warnings; or modified or repaired by unauthorized persons; or improperly installed. UNDER NO CIRCUMSTANCES SHALL EXCELSIOR BE LIABLE FOR ANY INCIDENTAL, INDIRECT, CONSEQUENTIAL OR SPECIAL DAMAGES, LOSSES OR EXPENSES (even if Excelsior had been advised of the possibility of such damages) arising from the sales contract or its performance or in connection with the use of; inability to use or repair of Excelsior's product for any purpose whatsoever. BUYER AGREES THAT ANY RECOVERY AGAINST EXCELSIOR SHALL NOT BE GREATER THAN THE PURCHASE PRICE PAID FOR THE PRODUCT AS TO WHICH SUCH CLAIM IS MADE. EXCELSIOR'S SELECTION TO ONE OF ITS ALTERNATES (REPLACE OR REPAIR) SHALL BE BUYER'S EXCLUSIVE AND SOLE REMEDY. No person is authorized to modify or amend the written warranty, product claims and specifications or to bind the corporation to any term contrary to the terms herein.

Service performed by persons other than Excelsior or its authorized agents, may, at the discretion of Excelsior, be cause to void this warranty. Excelsior's tubing sets are the only tubing products that may be used with the Pharm-Assist. If other sets are used, it may, at the discretion of Excelsior, be cause to void this warranty.

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