

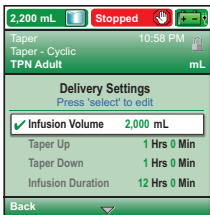
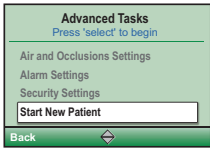


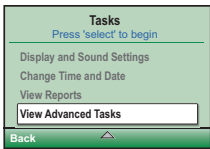

CADD-Solis™ Infusion System CADD-Solis VIP™ Ambulatory Pump Taper Mode

Programming a New Patient

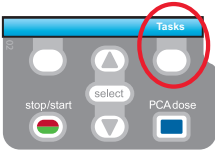

1. To start programming, select **Tasks**.
 2. Press **▼** to highlight **View Advanced Tasks** + **select**.
 3. Using the **▼** key, highlight **Start New Patient** + **select**. Then select **Continue**.
 4. Enter the 3-digit security code using **▼** or **▲** keys + **select**.
 5. Using the **▼** or **▲** keys, highlight **Taper** + **select**. Highlight **[Manual Progr]** or protocol qualifier if existing + **select**. If protocol, highlight drug name + **select**.
 6. Select **Yes** to confirm selection. Select **Review** to **Review Pump Settings**.
 7. Next, adjust all of the settings. Highlight a setting, press **select** then **▼** or **▲** then select **Save**.
- Reminder**
Infusion Volume: total volume of fluid to be delivered
Reservoir Volume: volume of fluid contained in the bag
8. Confirm values by selecting **Accept Value**, the symbol **✓** appears on the left + **Next**.
 9. Remove the blue clip and attach the tubing. The pump will show the next steps to follow until pump start.
 10. Start the pump.

*Taper Mode: commonly used for parenteral nutrition

Quick Change of Disposable

Same Infusion Volume, Same Programming



Power Button



1. Connect the CADD™ administration set to the bag. Without removing the blue clip on the cassette, prime the tubing manually until all air is removed. Or follow instructions below to prime using the pump.
2. Turn on the pump.
3. Remove the blue clip and attach the new disposable to the pump.
4. The pump displays the question **Reset reservoir volume to XX mL?** Select **YES** to confirm.
5. The screen displays **Prime tubing?** Select **YES** to prime using the pump. Select **No** if tubing was pre-primed.
6. Enter the 3-digit security code. Press **▼** or **▲** + **select** if needed.
7. Start the pump.