





CADD-Solis™ Infusion System CADD-Solis VIP™ Ambulatory Pump

PCA Mode


Programming a New Patient

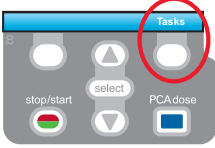


1. To start programming, select **Tasks**.
2. Press **▼** to highlight **View Advanced Tasks** + **select**.
3. Using the **▼** key, highlight **Start New Patient** + **select**. Then select **Continue**.

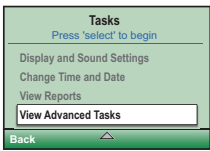

4. Enter the 3-digit security code using **▼** or **▲** keys + **select**.
5. Using the **▼** or **▲** keys, highlight **PCA** + **select**.

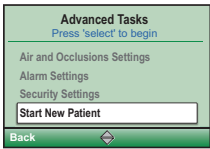


- Highlight **[Program Manually]** or protocol qualifier if existing + **select**. If protocol, highlight drug name + **select**. If Program Manually, highlight unit (mL, Mg, mcg) + **select**; enter Concentration value + **select**.
6. Select **Yes** to confirm selection. Select **Review** to **Review Pump Settings**.

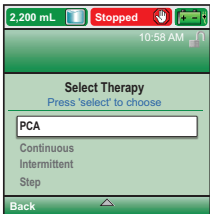

7. Next, adjust all of the settings. Highlight a setting, press **select** then **▼** or **▲** then select **Save**.

If a Delivery Limit or Max Doses/Hour is required and not available in the parameter list, select View Advanced Tasks then Delivery Hard and Soft Limits then Delivery Limit Method.

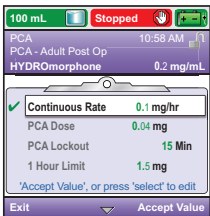

8. Confirm values by selecting **Accept Value**, the symbol **✓** appears on the left + **Next**.
9. Remove the blue clip and attach the administration set or the cassette. The pump will show the next steps to follow until pump start.
10. Start the pump.

Quick Change of Disposable

Same Infusion Volume, Same Programming



1. Remove the blue clip and attach the new disposable.
2. The pump displays the question **Reset reservoir volume to XX mL?** Press **YES** to confirm.
3. The screen displays **Prime tubing?** Select **YES** (if needed).
4. Enter the 3-digit security code. Press **▼** or **▲** + **select** if needed.
5. Follow the instruction to prime the tubing.
6. Start the pump.