




# CADD-Solis™ Infusion System CADD-Solis VIP™ Ambulatory Pump Continuous Mode


## Programming a New Patient




1. To start programming, select **Tasks**.

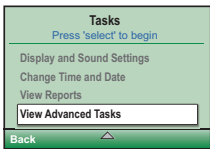

2. Press **▼** to highlight **View Advanced Tasks** + **select**.

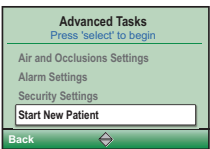


3. Using the **▼** key, highlight **Start New Patient** + **select**.  
Then select **Continue**.

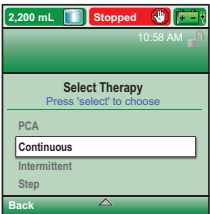

4. Enter the 3-digit security code **▼** or **▲** + **select**.

5. Using the **▼** or **▲** keys, highlight **Continuous** + **select**.  
Highlight **[Manual Progr]** or protocol qualifier if existing + **select**.  
If protocol, highlight drug name + **select**.

6. Select **Yes** to confirm selection. Select **Review** to **Review Pump Settings**.

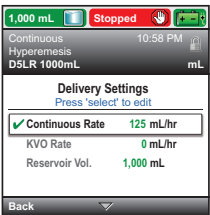

7. Next, adjust all of the settings. Highlight a setting, press **select** then **▼** or **▲** then select **Save**.

8. Confirm values by selecting **Accept Value**, the symbol **✓** appears on the left + **Next**.




9. Remove the blue clip and attach the administration set or the cassette. The pump will show the next steps to follow until pump start.

10. Start the pump.

# Quick Change of Disposable

## Same Infusion Volume, Same Programming



1. Turn on the pump.
2. Remove the blue clip and attach the new disposable to the pump.
3. The pump displays the question **Reset reservoir volume to XX mL?** Select **YES** to confirm.
4. The screen displays **Prime tubing?** Select **YES** to prime using the pump. Select **No** if tubing was pre-primed.
5. Enter the 3-digit security code. Press  or  +  if needed.
6. Follow the instruction to prime the tubing.
7. Start the pump.