








CADD-Solis™ Infusion System

CADD-Solis VIP™ Ambulatory Pump

Intermittent Mode

Programming a New Patient

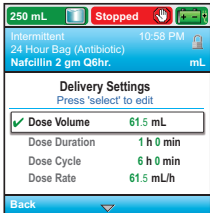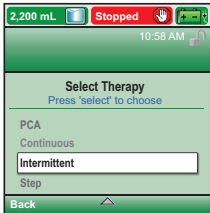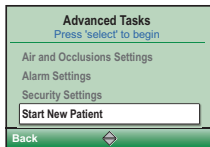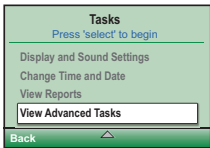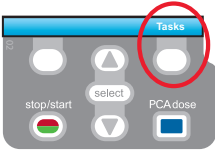



1. To start programming, select **Tasks**.
2. Press **▼** to highlight **View Advanced Tasks** + **select**.
3. Using the **▼** key, highlight **Start New Patient** + **select**. Then select **Continue**.
4. Enter the 3-digit security code using **▼** or **▲** keys + **select**.
5. Using the **▼** or **▲** keys, highlight **Intermittent** + **select**. Highlight **[Manual Progr]** or protocol qualifier if existing + **select**. If protocol, highlight drug name + **select**.
6. Select **Yes** to confirm selection. Select **Review** to **Review Pump Settings**.
7. Next, adjust all of the settings. Highlight a setting, press **select** then **▼** or **▲** then select **Save**.

Dose Volume:	Dose volume to be delivered
Dose Duration:	Time required to deliver a dose.
Dose Cycle:	Time from the start of one dose to the start of the next dose.
Next Dose:	Indicates the time at which the next dose will start.
8. Confirm values by selecting **Accept Value**, the symbol **✓** appears on the left + **Next**.
9. Remove the blue clip and attach the administration set or the cassette. The pump will show the next steps to follow until pump start.
10. Start the pump.

Quick Change of Disposable

Same Infusion Volume, Same Programming



1. Turn on the pump.
2. Remove the blue clip and attach the new disposable to the pump.
3. The pump displays the question **Reset reservoir volume to XX mL?** Select **YES** to confirm.
4. The screen displays **Prime tubing?** Select **YES** to prime using the pump. Select **No** if tubing was pre-primed.
5. Enter the 3-digit security code. Press  or  + **select** if needed.
6. Follow the instruction to prime the tubing.
7. Start the pump.