fra

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eng

Instructions for Use Tamper Evident Cap with Male Luer Lock

Indications for use

The Tamper Evident Cap with Male Luer Lock is indicated for use as a positive closure cap with evidence of access on female luer connectors on medical devices such as containers and sets.

Operating Instructions

Use Aseptic Technique

- Hold package firmly at tab and peel open while maintaining aseptic technique.
- Insert the fluid administration line female luer connector into the cap and rotate clockwise while pushing downwards.
- Remove Tamper Evident Cap with Male Luer Lock from package with fluid administration female luer connector installed.
Verify the installation by rotating luer connectors until snug
- To Remove the Male Tamper Evident Cap, grip the female luer at its base (not the tubing, if present) and pull the Male Tamper Evident Sleeve straight off.
- Twist off the tip cap counter clockwise.
- Remove the small ring from the female luer connector and properly dispose of sleeve, cap and ring.

Contraindications

None when used as described in the device's Instructions for Use.

Caution

- Damaged or missing Tamper Evident Sleeve indicates potential tampering and medication should be discarded.
- Do not resterilize.
- Single Use only, do not reuse, do not reapply Luer Lock cap onto female luer connector after cap is removed.
- Properly installed Luer Lock Cap remains sealed on connector until removed.
- Clamp or close fluid line prior to connecting/disconnecting the Tamper Evident Cap.
- Ensure all fluid administration line connections are secure prior to use.
- Not intended for use with Luer activated valves, Luer activated devices, and needle free IV connections.

STERILE EO	Sterilized using Ethylene Oxide
	Do not use if package is damaged
	Do not re-use
	Consult instructions for use
Rx ONLY	Federal (USA) law restricts this device to sale by or on the order of a physician

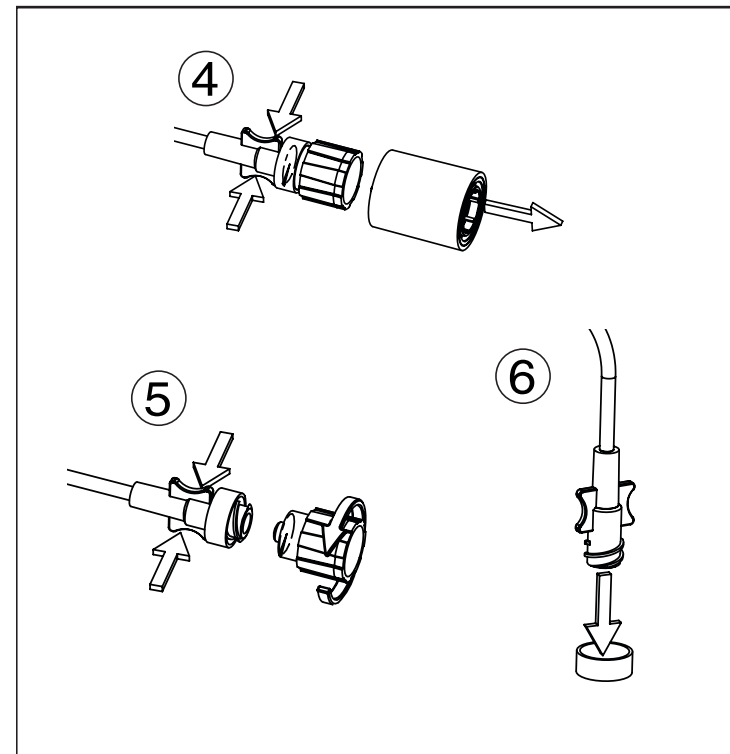
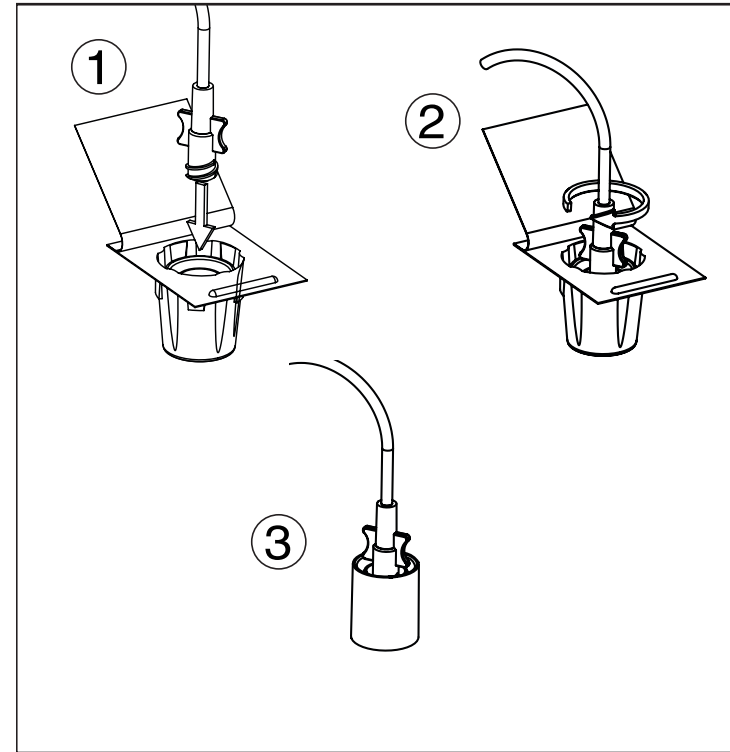
	Use-by date
	Manufacturer
	Non-Pyrogenic
MADE IN USA	

This product may be covered by one or more of the following U.S. Patents: U.S. 8,864,707; U.S. 8,864,708

International Medical Industries, Inc.
2981 Gateway Drive
Pompano Beach Florida, 33069 USA
www.imiweb.com, (954) 917-9570

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