



Universal 6" Flex Tube

Connector: 15F/22F

DESCRIPTION

The 6" flex tube is a disposable single use 15F/22F catheter mount for flexible connection to the patient

INSTRUCTIONS FOR USE

1. To connect to a T-Piece closed suction catheter; attach the 15F side to the step-down closed suction adaptor and then attach the 22F side to the HME/Breathing Circuit.
2. To connect to a Double Swivel Elbow (DSE) closed suction catheter; attach the 15F side to the swivel elbow and then attach the 22F side to the HME/Breathing Circuit.

WARNINGS

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- Do not soak, rinse, wash or sterilize this product.

TECHNICAL DATA

Dead Space	40 mL
Weight	9.5 g
Connections	15F/22F

SYMBOLS

SYMBOL	SYMBOL NOTES
	Consult instructions for use
	Cautions and Warnings
	Keep dry
	Keep away from heat and sunlight
	Do not re-use
	Do not use if package is damaged
	Not made with natural rubber latex
	Not Manufactured with di(2-ethylhexyl)phthalate
	Batch code
	Prescription use only



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php