

Infinity[∞]orange[®]

Small Volume
Enteral Feeding Pump



An enteral-only feeding system
optimized for small-volume
feedings and increased safety.





The Infinity Orange is an enteral only feeding system that features maximum programmable rate and dose lock features. The pump requires rate, dose, and food type confirmation before running, reducing the **potential for programming errors**.



The Infinity Orange maintains $\pm 5\%$ accuracy, even at low rate and dose settings. The pump **leaves minimal fluid in the downstream tubing after feeding completion, resulting in less wasted milk or formula.**



The Infinity Orange features an **intuitive design and interface** that can be quickly and easily mastered, allowing the caregiver to focus on other areas of care.



The Infinity Orange has been **proven to perform consistently** under a wide variety of conditions. The pump does not require any regularly scheduled maintenance or calibration and can be cleaned with warm soapy water.

Nutrition delivery optimized for small volume feeding

The **Infinity Orange Small Volume Enteral Feeding Pump by Moog** is an enteral only feeding system that is optimized for small volume feeding. It is powerful and accurate enough to use in the hospital, yet lightweight and portable enough to be used in the home.


The Infinity orange features an intuitive design and interface and is very simple to program, reducing the potential for programming errors.

Two convenient delivery set options can be used with either milk or formula: a 100 mL Bag Set, or a Screw Cap Delivery Set.

Safety and convenience built into every set

- The Infinity Orange Pump and striped tubing sets are easily identified as enteral feeding components, even when used in crowded hospital environments.
- Infinity Orange delivery sets are not manufactured with DEHP or natural rubber latex, and feature the ENFit® connector system with transitional stepped connector to help to further reduce the possibility of misconnections.
- A patented in-line occluder provides automatic free-flow protection in every set.

A uniquely designed delivery cassette and molded tubing segment enable pumping accuracy no matter the flow rate and operating orientation.

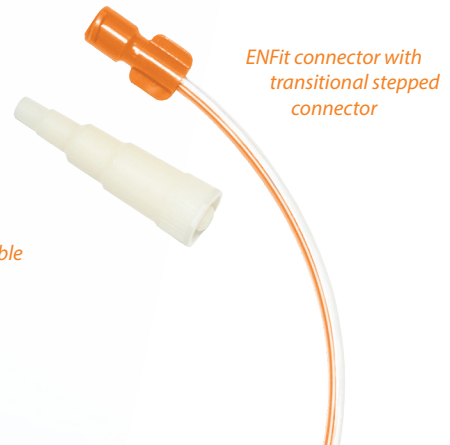

Infinity Orange Pump Specifications

Size:	4.05" H x 5.65" W x 1.95" D (10.3 cm H x 14.4 cm W x 4.9 cm D)
Weight:	14.4 oz (408 g)
Flow Rate Range:	0.1 – 300 mL/hr
Flow Rate Accuracy:	± 5%
Flow Rate Increment:	0.1 mL/hr from 0.1 – 10 mL/hr; 1 mL/hr from 10 – 300 mL/hr
Dose Range:	0.1 mL – 3000 mL
Dose Increment:	0.1 mL from 0.1 10 mL; 1 mL from 10 – 3000 mL
Priming Volume:	~5.5 mL
Residual Volume:	~2.0 mL
Battery:	Lithium ion; 24 hours at 125 mL/hr; 6 hour recharge time
Operating Orientation:	Any
Maintenance:	No annual maintenance or calibration required
Cleaning:	Washable under running water (water resistant)
Warranty:	2 years

Ordering Information

Order No.	Units / Box	Description
INFORNG	1	Infinity Orange Small Volume Enteral Feeding Pump Kit - includes Infinity Orange Pump, Compact Wall Charger, Adjustable Angle Pole Clamp, and operator's manual
INF0100-A	30	Infinity Orange 100 mL Bag Set
INFOCAP-A	30	Infinity Orange Screw Cap Delivery Set
23401-001	1	Compact Wall Charger
29152-001	1	Adjustable Angle Pole Clamp
PCK1001	1	Mini Backpack – Red
PCK1002	1	Mini Backpack – Light green and gray
PCK1003	1	Mini Backpack – Black and gray
PCK2001	1	Super-Mini Backpack – Black and gray
PCK2002	1	Super-Mini Backpack – Light green and gray
PCK3001	1	Waist Pack – Black and gray
PCK4001	1	Backpack – Black and gray

Note: Teal colored Infinity Delivery Sets can be used with the Infinity Orange Pump.



1) Infinity 100 mL Bag Set incorporates a built-in hanger, and large top-fill opening with leak-proof cap to minimize spills and waste.

2) Screw Cap Sets are compatible with all bottles and containers with a 40 mm opening. These sets are pre-assembled with a sleeve to allow bottle to be hung from an IV pole.



3) Adjustable Angle Pole Clamp, 4) Compact Wall Charger, 5) Carry packs

Industry-leading clinical and customer support options at your fingertips.



Live Clinical & Customer Support

Speak live with a Moog Clinical Representative for pump questions and troubleshooting guidance 24 hours per day, 7 days per week. Our Customer Support Team is also available Monday through Friday, 7:00 AM - 5:00 PM (Mountain Time) for customer service assistance such as product line questions, orders, complaints, and RMA support. Call **800.970.2337** for live clinical or customer support.



Live Stream Training

The Moog Clinical Team provides a variety of classes and custom sessions conveniently available through live stream video. Our Clinical Specialist will connect with your team in real-time direct from our office.

e-Learning

Become familiar with our pump technologies from any device in any location, all at your own pace! This comprehensive program covers all aspects of programming and troubleshooting each of our pumps. Customers also have the ability to administer e-Learning to their own employees. Register new employees as they are hired, track all training progress for your facility, and run reports to evaluate education compliance.



On-Site Education

For larger groups, our Clinical Team can work with your administrator to organize and develop in-person training at a location of your choice. For on-site education requests, please contact your sales representative or call our Clinical Team at **800.970.2337**.



Online Training Materials

The **infinityfeedingpump.com** website offers a variety of educational resource materials, such as:

Training Videos:

Comprehensive training videos provide an overview of pump programming and troubleshooting.

Resource Library:

A collection of resources available for download, including quick reference guides, troubleshooting guides, operator's manuals, etc.

Virtual Pump:

Take an interactive guided tour of our pumps and practices cases.

Recognized for its innovative technology and small size, the Infinity® Enteral Feeding Pump received the Medical Design Excellence Award and the Seal of Acceptance from the Children's Health Corporation of America. The Infinity Orange builds on the same platform, and offers many of the same clinical advantages.

Take an interactive tour of the Infinity Pump at infinityfeedingpump.com. Or, call us at **800.970.2337** to schedule a demonstration or request additional information.



CHILDREN'S HEALTH
CORPORATION OF AMERICA

Seal of Acceptance



Speak live with a Moog Clinical Representative for pump questions and troubleshooting guidance 24 hours per day, 7 days per week.

Clinical & Customer Support **800.970.2337**

Visit infinityfeedingpump.com for additional information



Tel: **801.264.1001**
Toll Free: **800.970.2337**
Fax: **801.264.1051**



4314 Zevex Park Lane
Salt Lake City, UT 84123 USA



infinityfeedingpump.com
medcustomerservice@moog.com



**carbon
neutral**
All Moog Administration and
Delivery Sets are manufactured
at a carbon neutral facility.

Infinity® and Infinity Orange® are registered trademarks of Zevex, Inc. Moog® is a registered trademark of Moog, Inc. ENFit® is a registered trademark of GEDSA. © 2019 Moog, Inc. Moog Medical, 4314 Zevex Park Lane, Salt Lake City, UT 84123 USA. MD-41259-001 Rev. D

MOOG®

Enhancing Healthcare.
Enriching Lives.™