





These instructions are not a substitute for the Operator's Manual. Be sure to read the Operator's Manual thoroughly prior to operating the pump.


### If priming manually for IV pole use

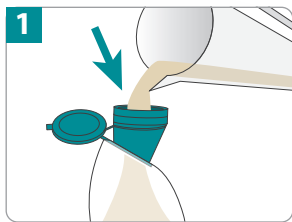


1 Hold the bag's funnel upright and pour in feeding formula. Close cap tightly.

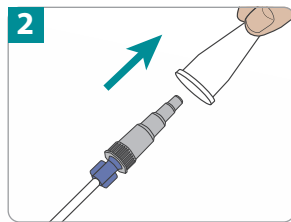

2 Remove protective cover from the feeding tube connector.

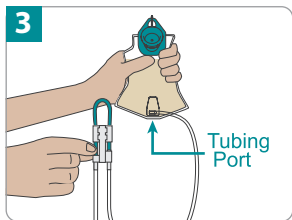


3 Prime the set manually by gently pinching teal colored tubing below the "O" symbol. While pinching, gently squeeze the bag at the same time to assist fluid flow. Hold this position until formula reaches the feeding tube connector.

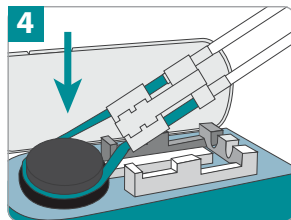


4 Loop the silicone tubing around the pump wheel stretching lightly. Seat cassette into the pump and close the door.

If using the pump in a backpack, refer to the Operator's Manual for detailed instructions for removing all air from the set.


### If priming on the pump

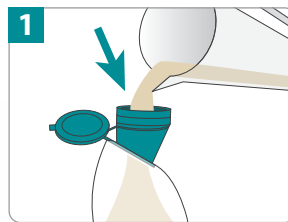


1 Hold the bag's funnel upright and pour in feeding formula. Close cap tightly and hang bag.

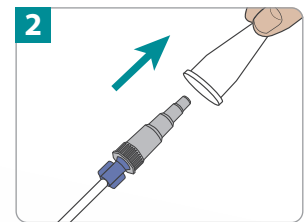


2 Remove protective cover from the feeding tube connector.

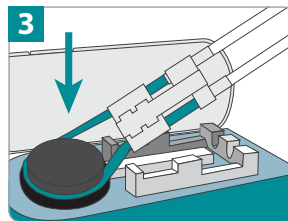

3 Loop the silicone tubing around the pump wheel stretching lightly.

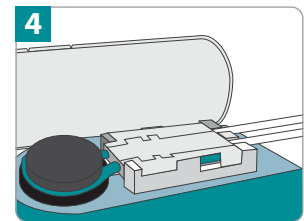

4 Seat cassette into the pump and close the door.

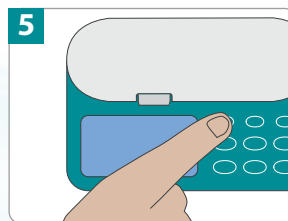

5 Turn pump on by pressing and holding the **ON/OFF** key until the pump beeps. Then, press and hold the **PRIME** key until formula reaches the feeding tube connector (approximately 90 seconds).



Speak live with a Moog Clinical Representative for pump questions and troubleshooting guidance 24 hours per day, 7 days per week.

Clinical & Customer Support **800.970.2337**  
Visit [infinityfeedingpump.com](http://infinityfeedingpump.com) for additional information

Infinity<sup>®</sup> is a registered trademark of Zeveex, Inc.  
Moog<sup>®</sup> is a registered trademark of Moog, Inc.  
© 2020 Moog, Inc. Moog Medical, 4314 Zeveex Park Lane,  
Salt Lake City, UT 84123 USA. LS-81153 Rev. A

**MOOG<sup>®</sup>**  
Enhancing Healthcare.  
Enriching Lives.™