


## NEOMED\* ORAL SYRINGE FOR ORAL/ENTERAL USE

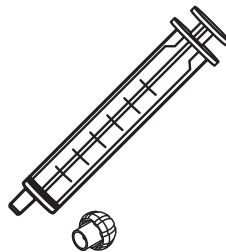

### Instructions for Use

#### Indications for Use

The device is indicated for use as a dispenser, a measuring device and an oral fluid transfer device. It is used to inject fluids into the body via extension sets and feeding tubes in neonatal and pediatric patients.

#### Contraindications

The NEOMED\* Oral Syringe is intended for one-time use. Each syringe should be used for one dispensing and one patient.

#### ⚠ Warnings

**Tip caps are a choking hazard. Keep out of reach of children.**

#### Instructions

1. Follow physician's instructions or hospital protocol in opening package and use of contents. Use aseptic technique.
2. Fully depress the plunger before filling oral syringe. Draw liquid (e.g. formula or mother's milk) into syringe. For accurate measurement, use the black leading edge of the gasket toward the plunger tip. The accuracy of measurements is +/- 4% of expelled volume. Do not fill and attempt delivery past the maximum graduation.

3. To expel excess air bubbles, hold syringe with tip pointing up. Tap side of syringe to cause bubbles to rise toward tip. Draw back slightly on plunger, then push plunger upward to eject air in barrel and tip. Repeat until bubbles are eliminated.
4. Place the tip cap on the syringe and label until ready to use (label not included).
5. Feeding should be administered via enteral-only connectors. This syringe is not designed for use with Luer connections. To minimize potential for clogging, mix any supplements or additives thoroughly to achieve uniform consistency.
6. Flush the syringe with water and dispose in closed plastics or dispose as required by hospital protocols and local or national regulations. Do not autoclave or re-use syringe as this may increase risk of leakage or biological infection.

Sterile
Size
3 ml
6 ml
12 ml
20 ml
35 ml
60 ml

MD Medical Device	No Recycled Plastic	STERILE/EO	Single sterile barrier system	Single sterile barrier system with protective packaging outside	No Glass
LATEX	BEPH	BPA	Rx Only	Warning	Information
Importer	UDI Unique device identifier				

Distributed in the USA by Avanos Medical Sales, LLC, 5405 Windward Parkway, Alpharetta, GA 30004 USA.

In USA, 1-844-4AVANOS (1-844-428-2667). [www.avanos.com](http://www.avanos.com)

Avanos Medical, Inc.,  
5405 Windward Parkway,  
Alpharetta, GA 30004 USA.

EC REP Avanos Medical Belgium BVBA,  
 Leonardo da Vincilaan 1,  
1930 Zaventem, Belgium.

Sponsored in Australia by Avanos Medical  
Australia Pty Ltd, 475 Victoria Avenue,  
Chatswood, NSW 2067, Australia.

CE  
2797

\*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates. © 2018 AVNS. All rights reserved. 2021-09-03 15-M1-1039-02