



Product Reference Guide

ChemoLock™

Needlefree Closed System Transfer Device (CSTD)

Oncology Kit (CL3960) for Ambulatory Cassette Preparation

1. Prepare Extension Set



- > Attach ChemoLock Extension Set in Kit to the CADD cassette.

2. Clamp Extension Set




- > Clamp extension set.

3. Infuse Drug



- > Refer to vial for syringe preparation instructions on how to prepare syringe.

4. Prime Line

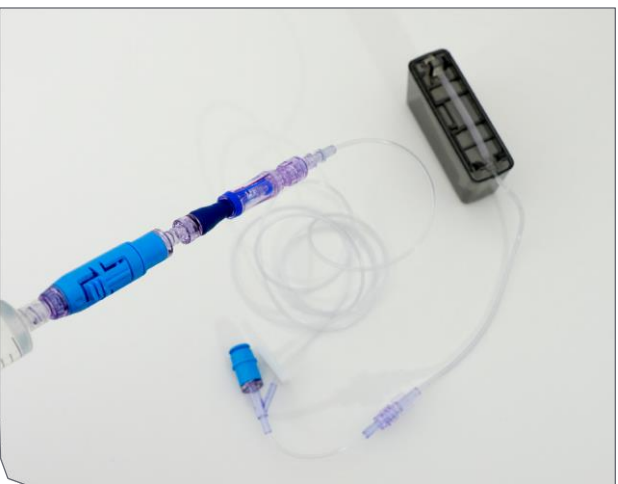

- > Prime using closed syringe method.
- > Remove red cap from Spiros.
- > Attach priming device to Spiros.
- > Pull fluid using Spiros or use pump to prime, otherwise according to protocol.

5. Disconnect



- > Disconnect Spiros from priming device after priming is complete.

6. Send to Nursing Floor



- > Place red cap and send to floor.

Questions:

Suzanne Williams
Oncology Specialist
Mobile: 704.614.2338

Chris MacNicol
Sales Representative
Mobile: 205-616-8218