


## Priming instructions between the 21-7361-24 and 21-7381-24.

Please note that the 21-7381-24 filter is an older filter and does take a little longer for it to become completely wet and primed. It is important to follow the instructions of the orientation of the filter.

21-7361-24 filter manufacturer: GVS

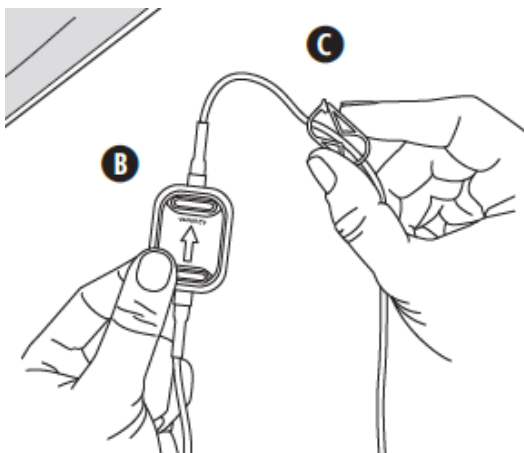


### **Gravity Priming:**

1. Close clamp. Remove bag spike cap and insert spike into IV bag. Hang bag. During priming hold filter with arrow pointing up. Open clamp and allow fluid to fill tubing and filter. Tap tubing as needed to remove any air. When filter and tubing are filled close clamp.

### **Pump Priming:**

1. Remove blue clip from cassette. Removing blue clip causes flow stop to close, preventing unintended gravity flow. Close clamp. Remove bag spike cap and insert spike into IV bag. Hang bag. Turn pump on. Attach cassette to pump and open clamp. During priming hold filter B with arrow pointing up. Prime tubing as instructed in *Operator Manual* (supplied with pump). Tap tubing as needed to remove any air. When filter and tubing are filled close clamp.

21-7381-24 filter manufacturer: PALL

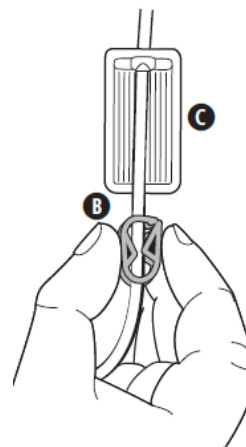

Allow filter to hang straight down.

**Gravity priming:** Open clamp and allow fluid to fill tubing and filter. Watch rib side of filter, making sure it fills evenly. Tap filter or tubing as required to remove any air. **NOTE:** You do not need to remove air from opposite side of filter (vent side). When rib side of filter and tubing are filled close clamp.

**Pump priming:** Remove blue clip from cassette attach cassette to pump and prime tubing as instructed in *Operator's Manual* (supplied with pump). When rib side of filter and tubing are filled close clamp.