
Usage of Oxygen (Accessory)

- O₂ must be turned off when the Vivo 45 LS is not in operating mode!
- It is recommended to monitor the oxygen • concentrations.
- 1. Mount the oxygen tube to the connector.



2. Connect the oxygen tube. A click sound is heard when the tube is properly attached.

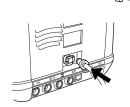

Disconnect:

<u> (</u>)

- 1. Turn off the oxygen source.
- 2. Press the release button to disconnect the oxygen tube.
- 3. Remove the tube.

Replacement of the Patient Air Inlet Filters

The filters are located in the filter

cassette.

Washable Filter (grey): Replace at least once a year, wash at least once a week.

Disposable Filter (white): Replace at least every 4th week, or more often in high pollution or pollenrich environments.



÷ -	Internal battery
┥╴	Click-in battery level
	Home Mode activated
b ^{22C} ≈	Humidifier and/or heated circuit connected
SP O2	SpO2 sensor connected
Fi O2	FiO2 connected
Et co2	EtCO2 connected
Ptc co2	PtcCO2 connected
	Effort belt connected
J•[00	Multiple pages
>>	Multiple content available

Vivo 45 LS

Quick Reference Guide Doc. 007726 En M-2 2022-11-30





Symbols

SYMBOL

DESCRIPTION

Breas Medical AB · Företagsvägen 1 SE-435 33 Mölnlycke · Sweden Phone +46 31 86 88 00 · Fax +46 31 86 88 10 Find your local distributor at www.breas.com






Ouick Reference Guide



This is not a complete guide for the Vivo 45 LS. See your manual for complete instructions.

Switch On/Off

1. Plug the power supply into the Vivo.

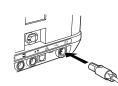

2. Press the Start/Stop button on the top panel. If a pre-use test is prompted, follow the instructions on the screen.





face.

Always perform a pre-use test when the patient circuit is replaced or modified.

3. Connect the other end of the tube to the patient inter-

For leakage circuits, the leak should be at least 12 l/min at


If the pre-use test is not prompted at start up, you can start it from the Others section.

- 1. Connect the patient circuit to the patient air outlet on the ventilator.
- 2. If having an active exhalation valve patient circuit, connect the pilot pressure tube at the back of the ventilator.



Start/Stop Treatment

1. To start treatment and enter operating mode first press and hold the Start/Stop button on the top panel.

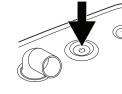

- 2. Release the Start/Stop button when the progress bar is filled.
- 3. To stop treatment and enter standby mode first press and hold the Start/Stop button on the front panel.
- 4. Release the Start/Stop button when the progress bar is filled.
- 5. Press the Audio Pause button to stop the treatment.

i-Button

Use the i-button to get explanations about settings and alarms.

Handle the Connectors

Make sure that the marking is facing up.

Pull the connector sleeve, not the cable itself to release the connector.



Connect the Click-in Battery (Accessory)

1. Release the side cover by pressing the button under the handle.



7Þ

2. Open and remove the side panel.

partment is engaged.



- 4. Close the side panel. Make sure there is a clicking sound to secure the side panel.







4 cmH2O to prevent rebreathing of exhaled air.





