

1



Recon Vial Spike (Dual Channel)

13 mm & 20 mm

2



Attach to vial. Hold the drug vial on a hard flat surface.

Push down until device snaps into place.

3




Remove IV Spike cover.

4




Insert into IV Bag Port.

5

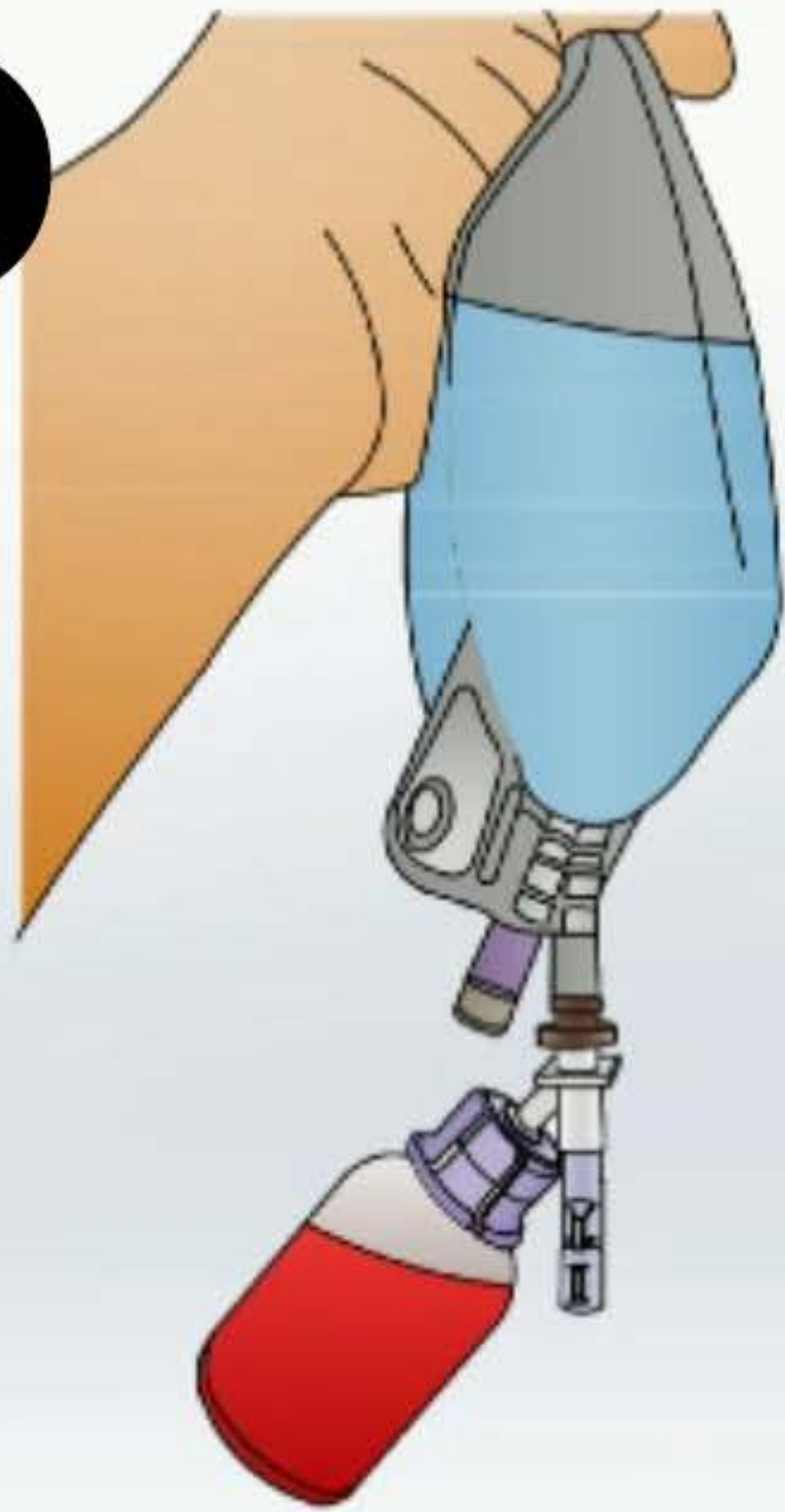

Reconstitute and mix. Squeeze bag to transfer liquid into vial.


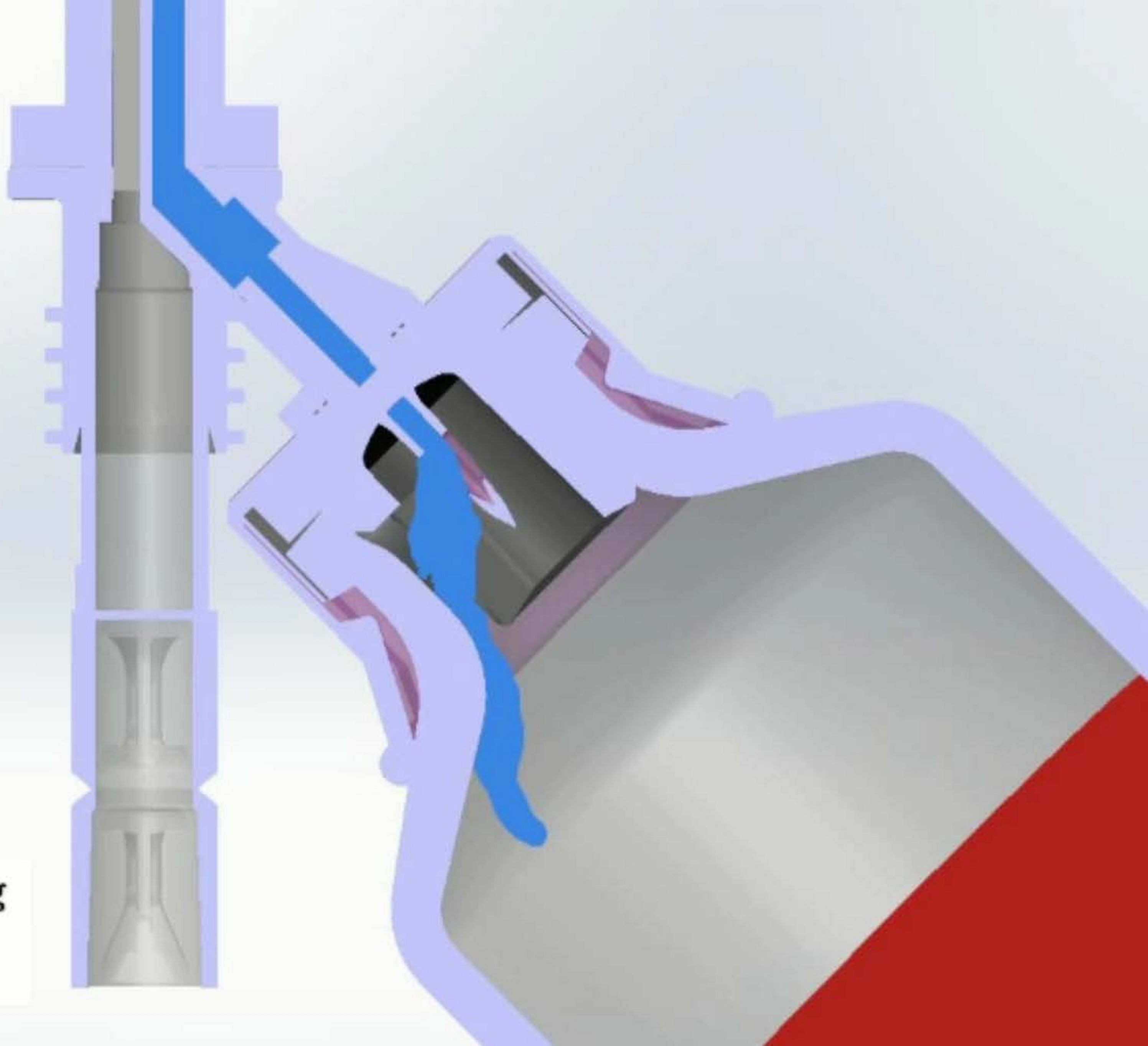


6




Mix with a gentle swirling motion.

7

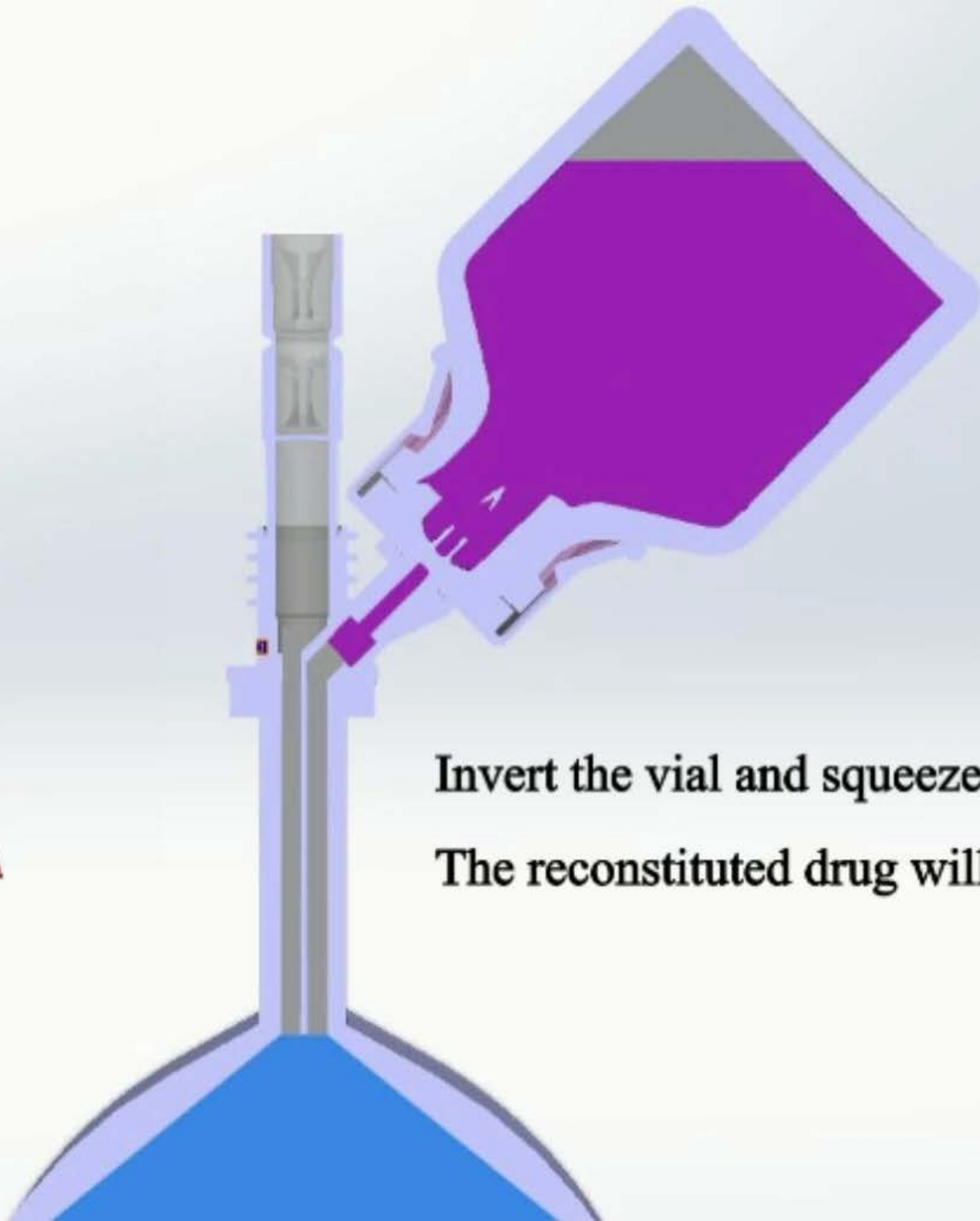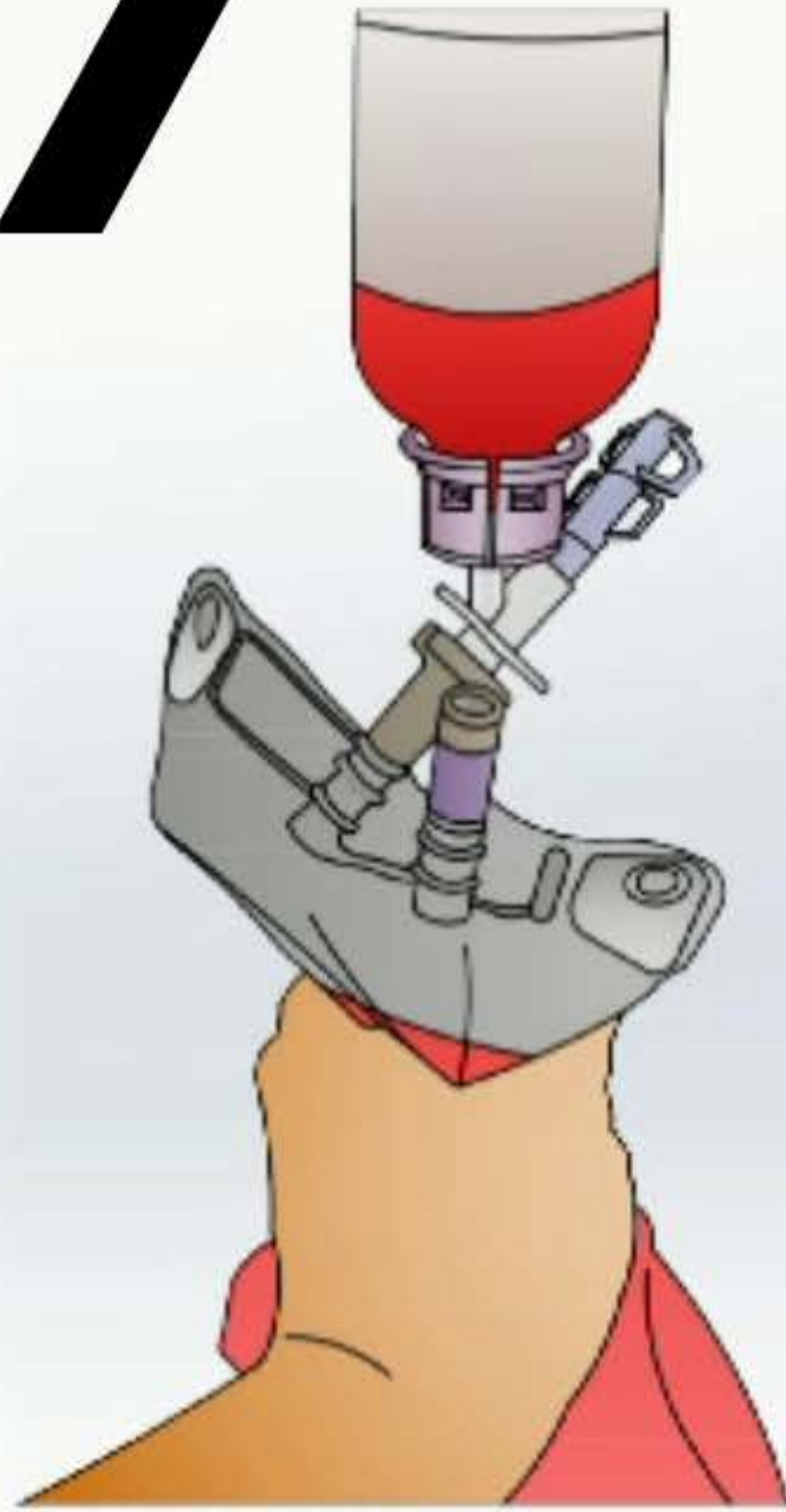

**Invert the vial and squeeze the IV Bag to push air into the vial.
The reconstituted drug will flow into the bag.**

8

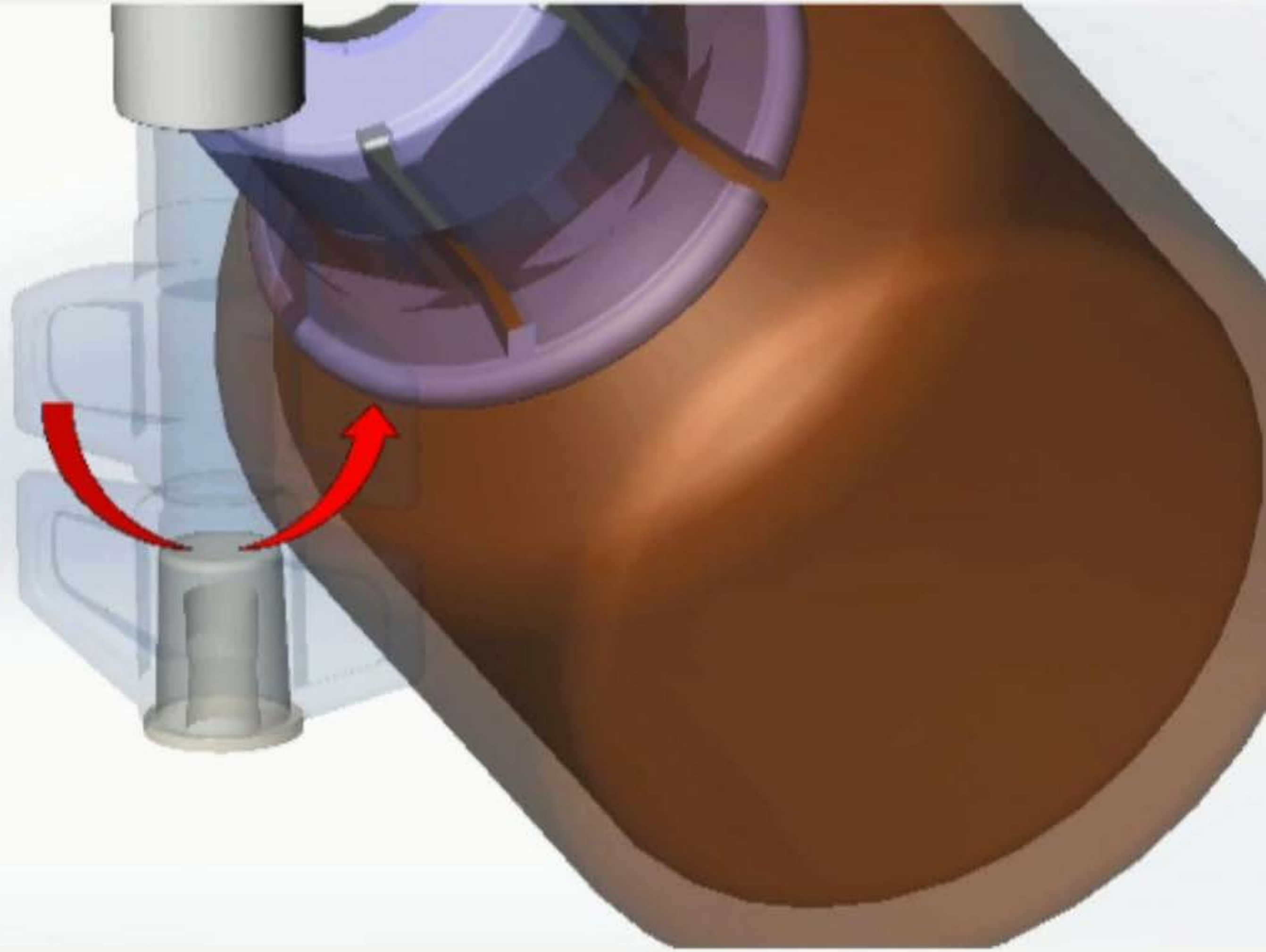

Open the IV port.

Insert IV Spike into IV port.

9

IV system is ready for administration.

Recon Vial Spike (Dual Channel)

