




Gravity Feeding Sets

Compatible with Ready-to-Hang Nutrition

Item Description	Item #	Units/Case
Screw Cap ENFit® w/ Transition Connector	VED-020	24
ENPlus Safety Screw w/ Transition Connector	VED-026	30

VED-020 VED-026

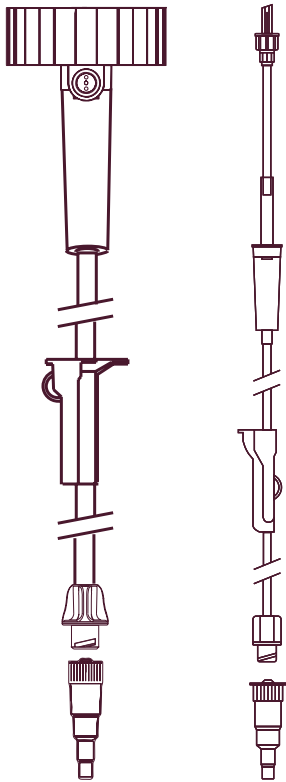

FEEDING RATES

Feeding Rate (mL/Hr)	Drops/Minute	Seconds Between Drops
25	6	10.3
30	7	8.6
50	12	5.1
65	15	4.0
75	18	3.4
100	23	2.6
150	35	1.7
200	47	1.3
250	58	1.0

Important Safety Information:

- To avoid risk of a tubing misconnection, ONLY connect the distal connector to an ENTERAL feeding tube. This device is compatible with a flexible female connector on the patient's enteral feeding tube. Not for IV use.
- This device should be replaced at least every 24 hours.
- Do not store for prolonged periods at extreme temperatures.
- For single-patient use only.
- Device is non sterile.
- See individual product labeling for directions.
- Rx Only

ENFit Transition Set:

- Pre-attached ENFit Transition Connector allows compatibility with existing feeding tube female ports.
- Removing ENFit Transition Connector allows compatibility with new ISO standard connection on new feeding tubes with ENFit male connectors.

